







**Invacare Action1R
MED3349**

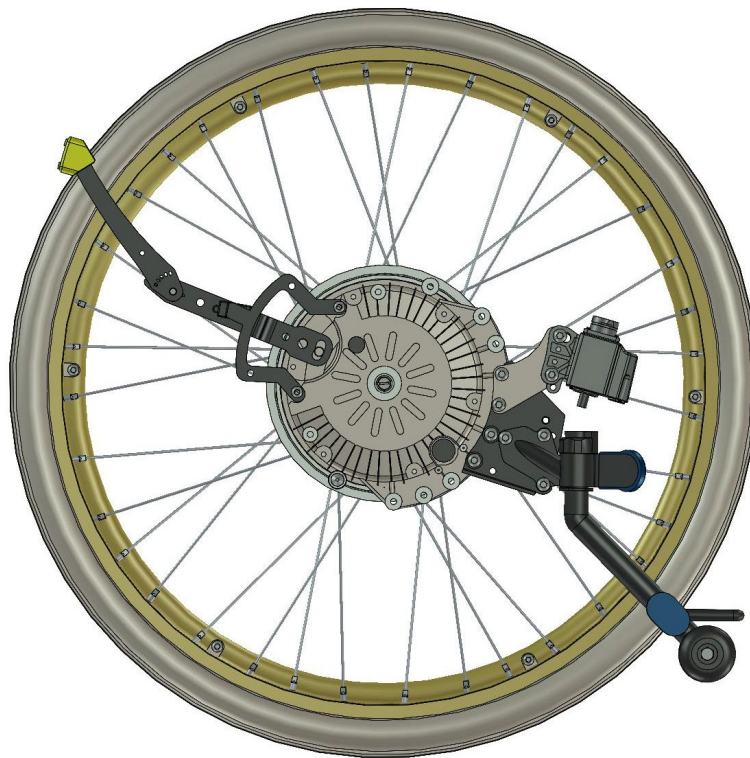

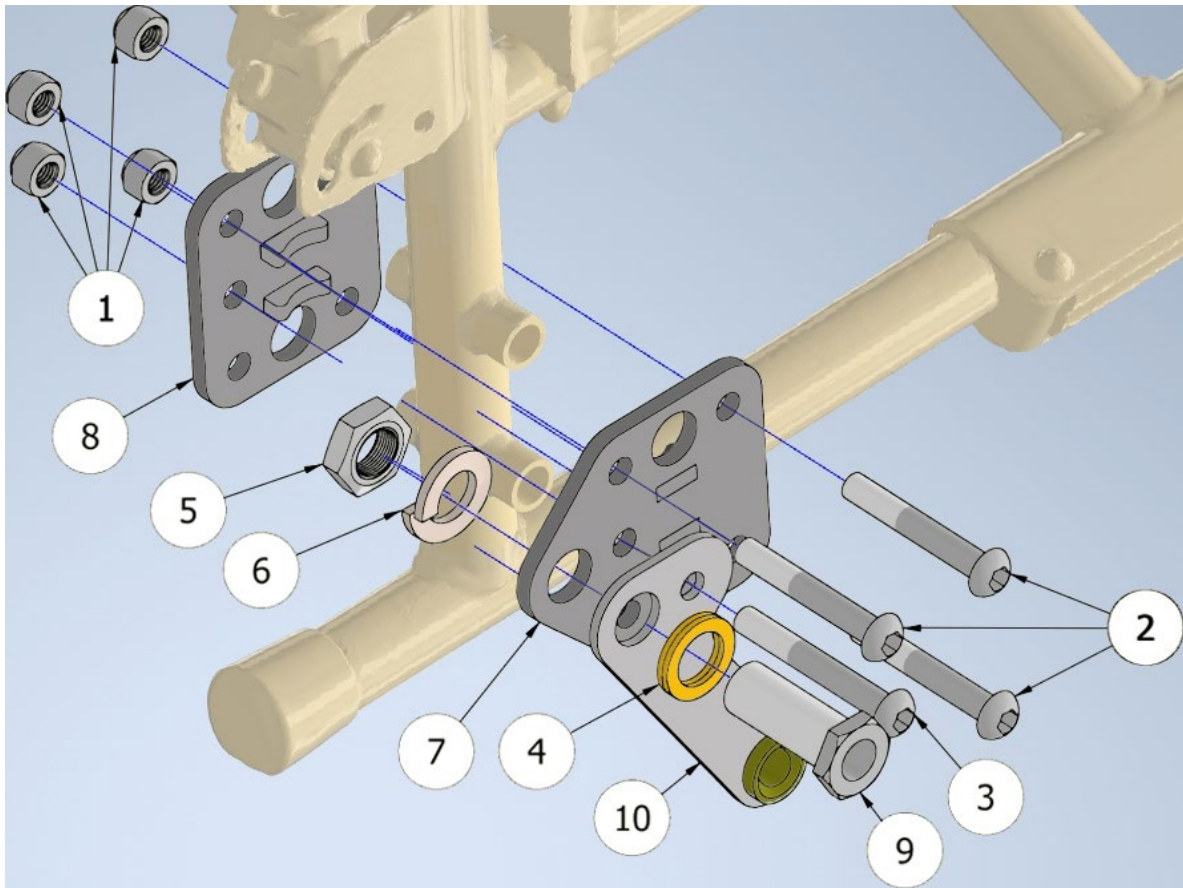


1	XSM8ML	4	
2	XSM8X55	3	
3	XSM8X60	1	
4	405239	1	
5	XSM16M	1	
6	XSM16FJ	1	
7	340026	1	
8	340027	1	
9	HW069	1	
10	MED2049	1	

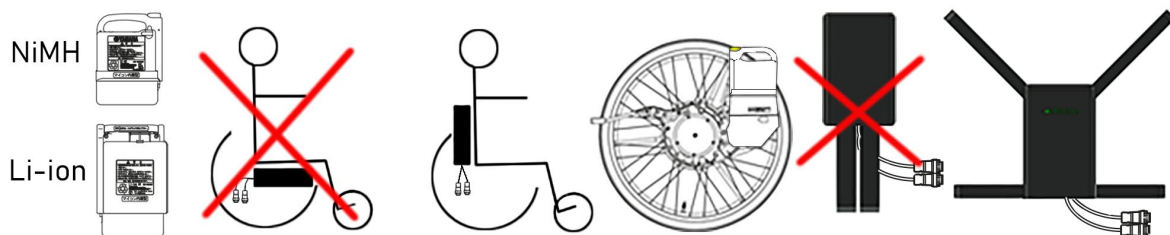

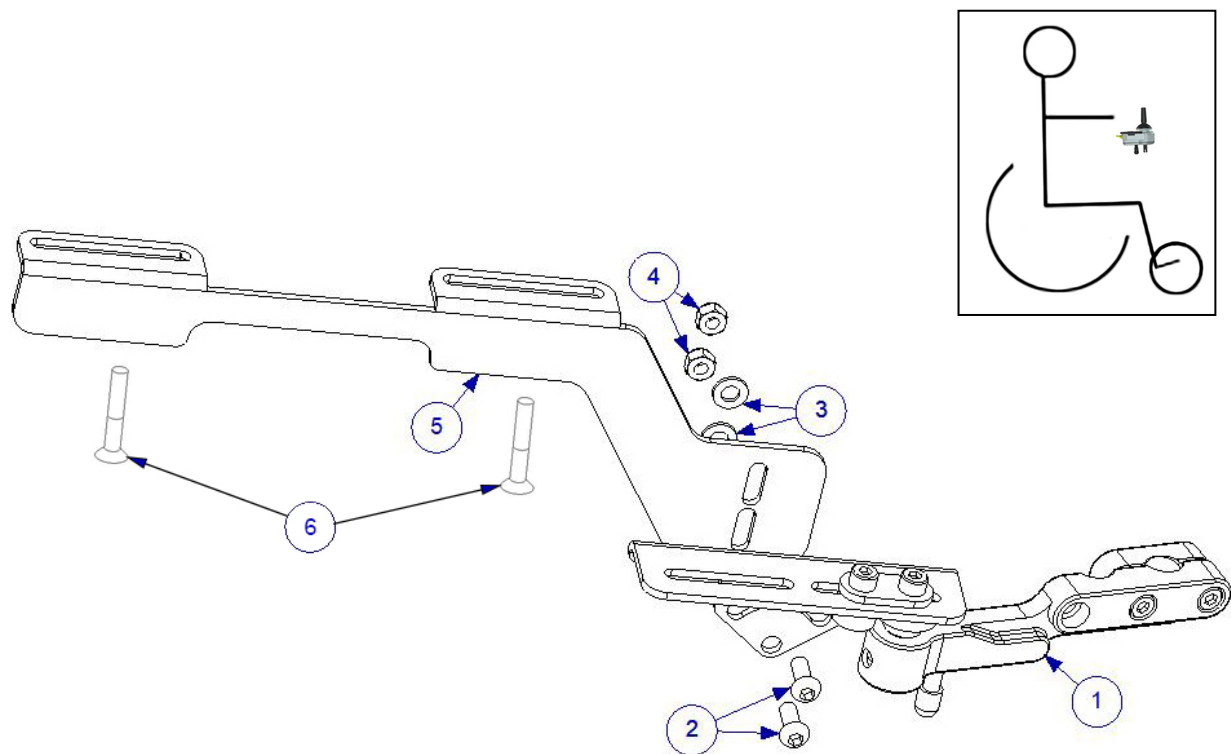


1	X0C-48160-B0	1	
2	XSM6X16	2	
3	XSM6BR	2	
4	XSM6ML	2	
5	MED1500 Rev6	1	
6	XSF5X30	2	

MED1356

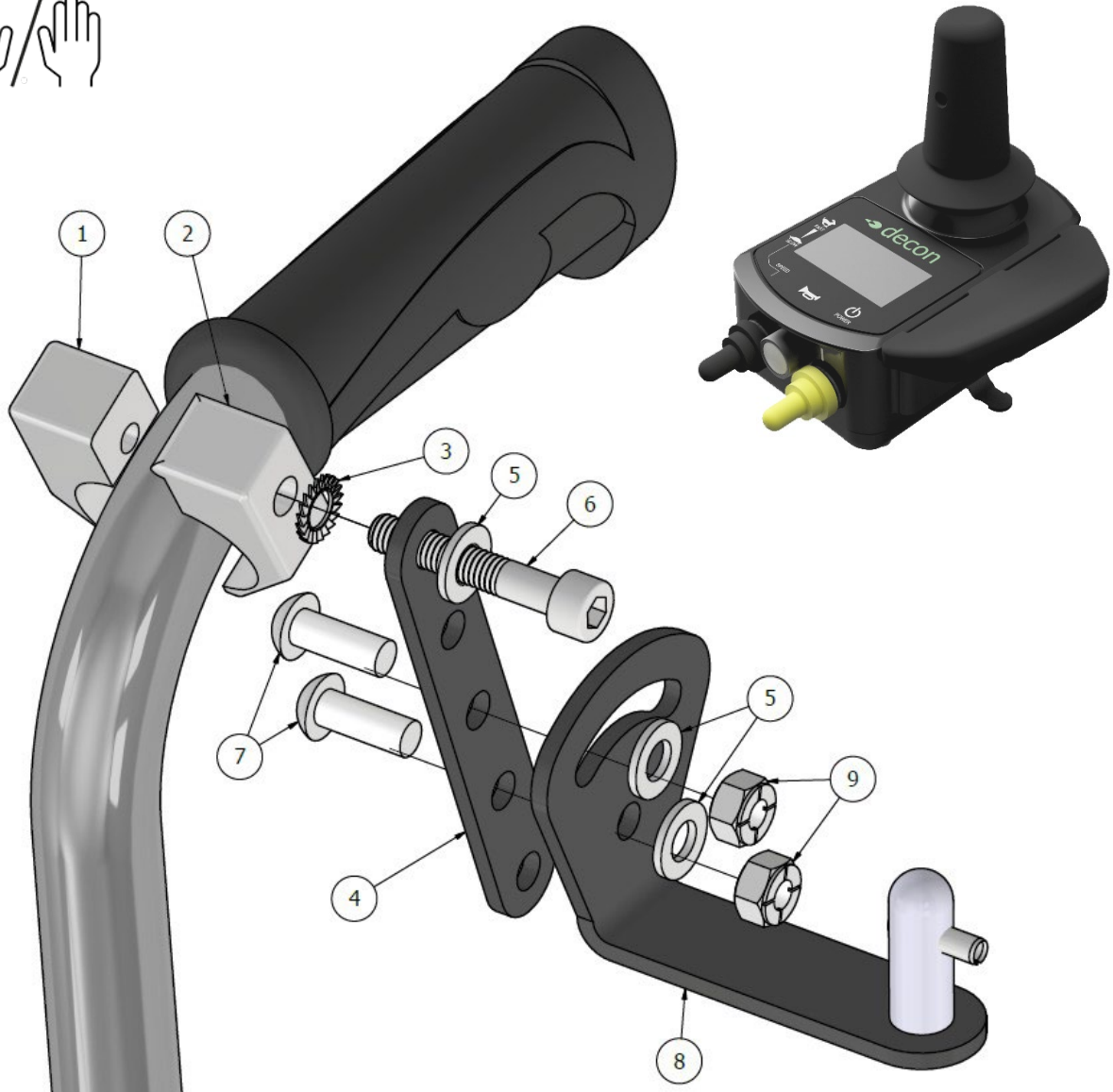

1	XA2-2612K-00	1	
2	XA2-2612E-00	1	
3	XSM6BRTG	1	
4	MED1317	1	
5	XSM6BR	3	
6	MC6SM6X35	1	
7	XSM6X16	2	
8	MED1316	1	
9	XSM6ML	2	

MED1218

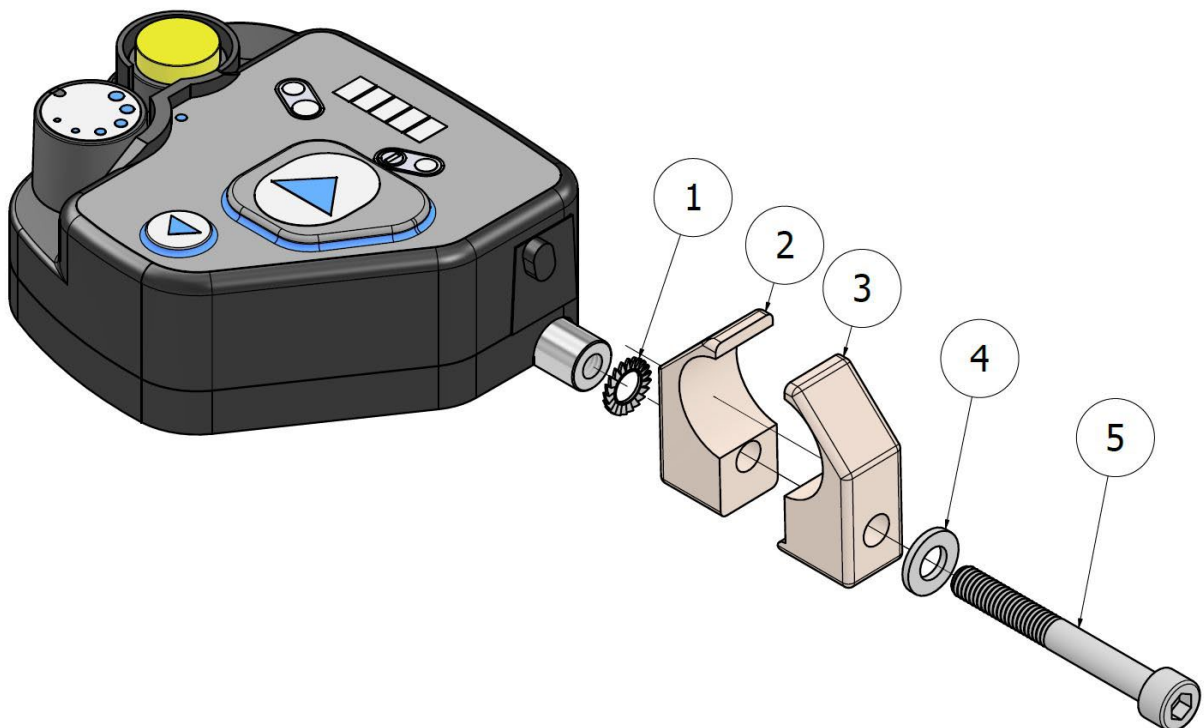


1	XSM6BRTG	1	
2	XOD-2612K-00	1	
3	XA2-2612E-00	1	
4	XSM6BR	1	
5	MC6SM6X50	1	



MED1218 + MED1643



M3	1,2 Nm
M4	3 Nm
M5	6 Nm
M6	10 Nm
M8	25 Nm
M10	50 Nm
M12	85 Nm

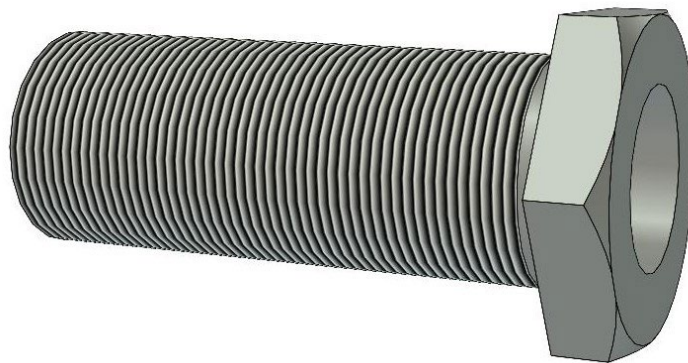

M16x1	30 Nm
M18x1	40 Nm