YOMPER +

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User Manual (EN – p. 24)

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Benutzerhandbuch (DE – p. 68)

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Manual do Utilizador (PT – p. 112)

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Brugervejledning (DA – p. 200)

acekare

**Fabricant**

ACEKARE
7 rue Mireport, 33310 Lormont
FRANCE
www.acekare.com
contact@acekare.com
+33 9.80.80.85.15

Version du manuel

Version : v2.0

Date de publication : 2026-03-20

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Version : v2.0

Publication Date : 2026-03-20

The information contained in this document is subject to change without notice. Please refer to the latest version on our official website.

**Follow the Instructions for Use**

The device has been designed and tested in accordance with applicable standards, including IEC 60601-1 relating to the safety of medical electrical equipment.

**Compliance Information**

This product complies with the applicable safety and performance requirements for medical devices under Regulation (EU) 2017/745.

The «CE» marking certifies its compliance with the safety and health standards required for marketing in Europe.

Copyright & Reproduction

© 2026 ACEKARE. All rights reserved.

Reproduction prohibited without authorization.

1 Introduction

- 1.1 Intended Purpose
- 1.2 Contraindications and Limitations of Use
- 1.3 General Safety Precautions
- 1.4 Specific Operating Conditions
- 1.5 Environmental Conditions
- 1.6 Compatibility and Modifications
- 1.7 Electromagnetic Interference (EMI)
- 1.8 List of components

p.26

p.26
p.26
p.26
p.27
p.27
p.27
p.28
p.28

2 Installation

- 2.1 Installing the Yomper mounting bracket
- 2.2 Installing the EasyGo control unit mount

p.29

p.29
p.29

3 Operation

- 3.1 Power-Up
- 3.2 Docking the Yomper to the wheelchair
- 3.3 Installing and connecting the EasyGo control unit
- 3.4 Controls and Modes
- 3.5 Powering Off and Standby Mode
- 3.6 Battery charging
- 3.7 Attendant throttle (Optional)

p.30

p.30
p.30
p.31
p.32
p.33
p.34
p.35

4 Functions & Settings

- 4.1 EasyGo control unit display
- 4.2 Settings Menu
- 4.3 Pairing

p.36

p.36
p.36
p.37

5 Maintenance

- 5.1 Inspections & preventive maintenance
- 5.2 Cleaning
- 5.3 Transport
- 5.4 Materials & Protection
- 5.5 Storage
- 5.6 Lithium battery Maintenance & Safety
- 5.7 Battery Removal
- 5.8 Reuse
- 5.9 Disposal

p.38

p.38
p.38
p.38
p.38
p.39
p.39
p.40
p.40
p.41

6 Troubleshooting

p.41

7 Warranty

p.43

- 7.1 Warranty Coverage
- 7.2 Warranty Exclusions
- 7.3 Limitation of Liability

p.43
p.43
p.43

8 Technical Specifications

p.44

- 8.1 Performance
- 8.2 Dimensions & Weight
- 8.3 Compatibility

p.44
p.45
p.45

1. Introduction

Welcome to the Yomper user manual. This document provides the information necessary to install, use and maintain the Yomper safely and in accordance with its intended purpose.

Read this manual carefully before use. Keep it for future reference.

Failure to follow these instructions may result in injury or loss of regulatory compliance of the device.

1.1 Intended Purpose

The Yomper is a medical device intended to assist the propulsion of a manual wheelchair by providing electrically powered assistance controlled by the wheelchair user.

It is intended for individuals with significant or substantial mobility limitations due to impairment of the lower limbs or conditions affecting walking ability.

The Yomper is designed for:

- indoor and outdoor use,
- operation within the environmental conditions specified in this manual,
- use with compatible manual wheelchairs.

The device is intended for users capable of:

- maneuvering a manual wheelchair (moving forward, turning, braking),
- understanding and safely operating the control functions of the Yomper.

An optional attendant control may allow operation by a third party under the responsibility of the wheelchair user.

The total weight (user + equipment) must not exceed the limits specified in the Technical Specifications section.

The installation of the Yomper mounting bracket must be carried out by a trained professional or by the manufacturer.

1.2 Contraindications and Limitations of Use

The Yomper must not be used in the following situations:

- inability to safely control a manual wheelchair,
- cognitive impairment preventing understanding of the controls,
- unauthorized modification of the device or the wheelchair,
- use with a non-compatible wheelchair,
- exceeding specified technical limits.

The device does not replace the braking capability of the wheelchair.

If there is any doubt regarding the user's ability to operate the device safely, consult your medical equipment provider.

1.3 General Safety Precautions

Before each use of the Yomper, the following precautions must be observed.

a. Familiarization

If you are using an electrically assisted propulsion system for the first time, practice in a safe environment (clear space, flat surface, no traffic).

Do not use the Yomper on public roads until you are fully confident with:

- acceleration,
- stopping,
- maneuvering (turning, reversing),
- speed control on slopes.

b. Anti-tip devices

The installation of anti-tip devices on the wheelchair is strongly recommended.

These devices reduce the risk of backward tipping, particularly:

- during acceleration,
- when ascending slopes,
- when negotiating obstacles.

If your wheelchair is not equipped with anti-tip devices, consult your dealer.

c. Pre-use checks

Before each journey:

- Ensure the battery is sufficiently charged.
- Verify that the Yomper is securely attached.
- Briefly test the control system (power on, throttle response).
- Check the general condition of the wheelchair (brakes, wheels, stability).
- Check the Yomper tire pressure: it must be properly inflated to ensure good performance (2.5 bar).

If any abnormal noise or malfunction occurs:

- stop using the device immediately,
- switch it off,
- contact your medical equipment provider.

1.4 Specific Operating Conditions

Use on slopes: Adjust your speed according to the incline. Maintain a stable position and keep control of the wheelchair at all times. When going uphill, lean slightly forward to maintain wheelchair stability and reduce the risk of tipping backward.

Obstacle negotiation: Switch off the device before negotiating significant obstacles (curbs, steps). Respect the wheelchair's obstacle limits.

Two-wheel maneuver (wheelie): The Yomper must be switched off before balancing on the rear wheels.

Use on uneven terrain: The Yomper can assist propulsion on non-paved surfaces, but mobility remains limited by the capabilities of the manual wheelchair. The small front wheels of the manual wheelchair may become blocked on uneven surfaces (holes, stones, gravel), which may lead to loss of stability and increase the risk of tipping. The Yomper does not increase the obstacle-crossing capabilities of the wheelchair. The user must adapt their speed and avoid obstacles that may block the front wheels.

1.5 Environmental Conditions

The Yomper is designed for indoor and outdoor use under normal environmental conditions.

Recommended operating conditions

The Yomper is intended to be used:

- on stable and flat surfaces,
- on hard surfaces such as concrete, asphalt, or indoor flooring,
- on regular surfaces without significant obstacles.

Conditions to avoid

The Yomper must not be used:

- in heavy rain,
- on icy or very slippery surfaces,
- on unstable terrain (mud, deep sand, loose gravel),
- in conditions exceeding the limits specified in the technical specifications.

Yomper performance may be reduced on irregular or loose surfaces.

1.6 Compatibility and Modifications

The Yomper must only be used with wheelchairs declared compatible.

Any unauthorized modification of the Yomper or the wheelchair:

- may create safety risks,
- voids the warranty,
- may invalidate regulatory compliance.

1.7 Electromagnetic Interference (EMI)

The Yomper complies with applicable electromagnetic compatibility requirements for medical devices.

However, strong radio transmission sources may cause:

- unintended activation,
- temporary malfunction,
- alteration of control.

The device has been tested up to an immunity level of 20 V/m.

To reduce risks:

- Avoid use near powerful transmitters.
- Do not add non-approved accessories.
- In case of unintended activation, switch off immediately and contact technical support.

1.8 List of components

Yomper Drive Unit

Aluminum frame	1
Brushless motor, 24V – 250W	2
Electronic controller	3
Power button (On/Off)	4
Carrying handle	5
Release lever	6
Attendant throttle connector	7
Battery charging port	8
Battery level indicator	9
Battery access cover	10
Battery charger	11
AC mains plug	12
Charging connector	13

EasyGo Control Unit

Main button (Power On/Off)	14
Speed control (+ / – depending on settings)	15/16
OLED display	17
Speed level / Mode indicator	18
On/Off button	19
Charging port	20
Magnetic charging cable	21
Mounting strap	22
Attendant control (optional throttle)	23

2. Installation

2.1 Installing the Yomper mounting bracket

The Yomper mounting bracket is a structural interface that secures the power unit beneath the wheelchair. It ensures safe and stable docking of the Yomper.


Each wheelchair type—**rigid frame** or **folding frame**—requires a specific mounting bracket design.


The installation of the Yomper bracket is critical to both the safety and performance of the system. It must only be carried out by an authorized medical equipment provider or at an official Acecare service center.

Do not attempt self-installation under any circumstances.

An incorrect installation may lead to equipment malfunction, loss of control, or user injury.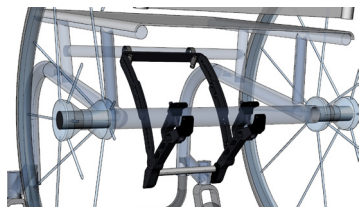

Example of bracket for **rigid frame** wheelchair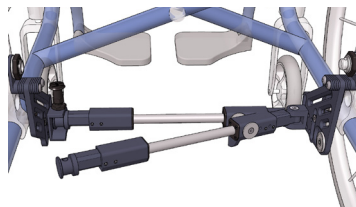


Example of bracket for **folding frame** wheelchair

2.2 Installing the EasyGo control unit mount

The EasyGo control unit must be securely mounted using the appropriate support. It can be installed either on the user's leg or directly on the wheelchair frame.


Leg mount installation

1. Ensure the strap is oriented correctly before securing it.
2. Clip the buckle and rotate the strap mount so the control holder is upright and stable.
3. Do not overtighten the strap: you should be able to insert a finger between the strap and your leg to avoid blood flow restriction.
4. Do not place the strap directly on bare skin. Always wear clothing or use padding to prevent skin irritation or discomfort.
5. Insert the EasyGo control unit into the mount, press down, and twist to lock it in place.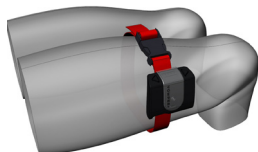


Wheelchair frame mount installation

1. Remove the thigh strap mount provided with the unit.
2. Position the frame bracket with the opening facing upwards.
3. Secure the bracket using the supplied cable ties, and place the rubber stabilizer to enhance grip and reduce vibration.
4. Tighten the bracket firmly onto the wheelchair tube, then trim the excess length of the cable ties.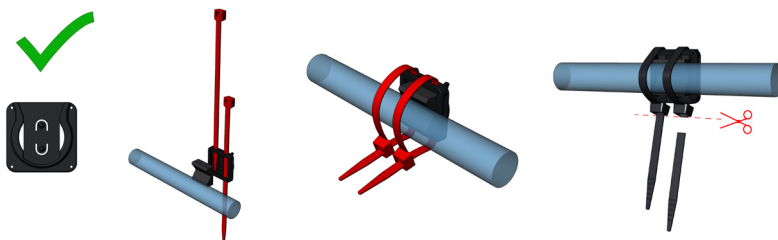

5. Insert the EasyGo control unit into the mount and twist to lock.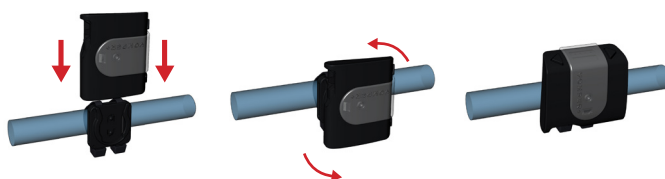

3. Operation

Before using the Yomper, carefully follow these steps to ensure safe operation.

3.1 Power-Up

1. Make sure the EasyGo control unit is switched off (screen and LEDs are off).
2. Press the POWER button on the Yomper.
3. A flashing LED indicates that the Yomper is waiting to connect with the EasyGo control unit.


3.2 Docking the Yomper to the wheelchair


- Place the Yomper on the ground behind the wheelchair and roll it forward.
- When the front coupling hook contacts the docking bar, push the transport handle down to engage the Yomper.
- Ensure the release lever is in the open position during this step.
- Once engaged, check that the Yomper is securely locked in place.

Warning: Improper docking may cause tipping or malfunction.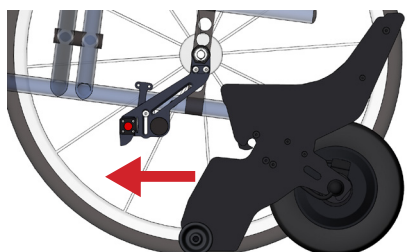


When the front coupling hook touches the docking bar, push the transport handle down to slide the Yomper onto the bar. Keep the release lever open during this operation.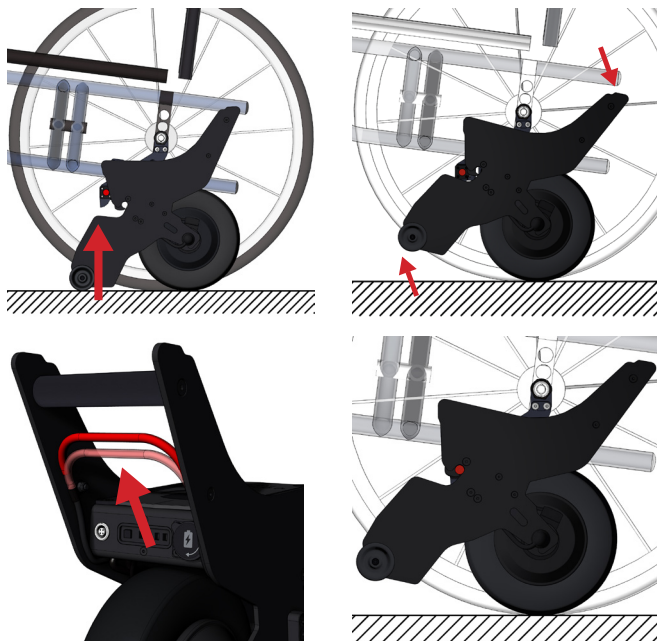


Once docked, ensure the Yomper is properly locked and secured.

3.3 Installing and connecting the EasyGo control unit

1. Mount the EasyGo control unit on its support (either on the leg strap or wheelchair frame).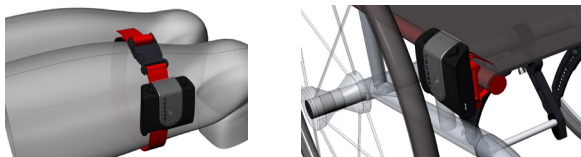

2. Switch on the EasyGo unit by pressing the ON/OFF button (19).

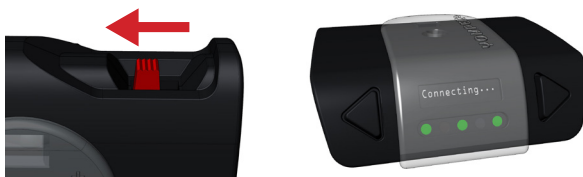During pairing, LEDs 1, 3, and 5 on the control unit will light up. Once the connection is established, the LED strip turns flashing green (Indoor Mode).

The EasyGo LED will blink, while the Yomper LED stays solid, indicating the system is ready to use.


3.4 Controls and Modes

Starting and Stopping the Yomper

Start: Double-press the **main button** on the EasyGo unit. The Yomper immediately starts in **Speed Level 1** (the EasyGo LED stops flashing).

Stop: Press the main button once. The Yomper motor disengages and switches to freewheel mode (the EasyGo LED flashes). You must use the wheelchair's push rims and brakes to slow down or stop.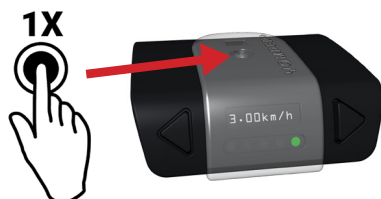

Warning: The Yomper does not have built-in brakes. Always use the wheelchair's push rims to prevent unintentional movement.

Speed Adjustment

Increase speed: Press the arrow button furthest from you (upper arrow).

Decrease speed: Press the arrow button closest to you (lower arrow).

Speed Indicator and Performance

The LEDs on the EasyGo unit show the selected speed level (up to 5 levels). However, the **actual propulsion speed** depends on several factors, including:

- Selected speed level (1 to 5).
- Operating mode: Indoor (green LED) or Outdoor (red LED).
- Custom Yomper settings (individual performance tuning).
- Environmental conditions: surface type, slope, obstacles.
- User and wheelchair total weight.
- Wheel type: solid tires or pneumatic tires, affecting traction and rolling resistance.


Keep in mind that the **actual speed may differ from the theoretical maximum speed** due to these parameters.

Switching Modes (Indoor / Outdoor)

To switch between modes, press and hold the main button for 2+ seconds until the LED strip changes color:

- From Indoor Mode (GREEN) -> switches to Outdoor Mode (RED).
- From Outdoor Mode (RED) -> switches back to Indoor Mode (GREEN).

Switching modes automatically resets the speed to Level 1.

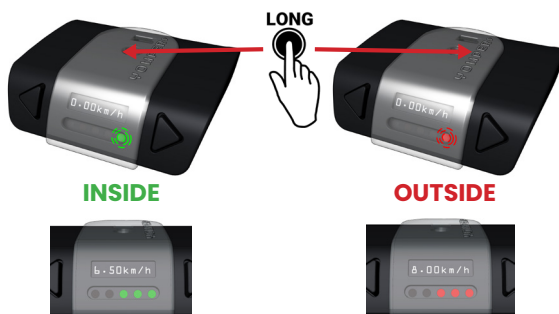In Indoor Mode (GREEN), you can also stop the Yomper motor by braking with the push rims.

In Outdoor Mode (RED), the motor can only be stopped by pressing the main button.

3.5 Powering Off and Standby Mode

After each use, always switch off the EasyGo control unit by pressing the ON/OFF button.

Once the control unit is switched off:

- The Yomper automatically enters **standby mode**, indicated by the rapid flashing of the **POWER button LED**.
- However, it is **strongly recommended** to fully power off the Yomper at the end of each day to preserve battery life and prevent unnecessary energy consumption.

Safety and Risk Prevention

- Always **switch off the EasyGo unit** when it is not in use to avoid unintentional motor activation or accidental wheelchair movement.
- During **storage or transport**, ensure the Yomper is **completely powered off** to prevent unintended start-up.

Battery preservation

- Turning off the Yomper completely each evening helps reduce premature battery wear and maintains long-term autonomy.
- A battery left under constant demand, even in standby mode, will lose capacity faster.

Never leave the Yomper powered on unnecessarily, especially when it will not be used for extended periods.

3.6 Battery charging

Yomper Battery Charging

A battery level indicator is located on the rear panel of the Yomper.

- Press the gauge button to check the battery charge level.
- The indicator is divided into 4 levels and provides an approximate charge status.

■	■	■	■		100 to 75%
■	■	■	□		75 to 50%
■	■	□	□		50 to 25%
■	□	□	□		Less than 25%

You can also check the Yomper's battery level more precisely using the EasyGo control unit (see Chapter 4 – Functions and Settings).

Charging Procedure

1. Switch off the Yomper using the POWER button before starting the charging process.
2. Connect the manufacturer-supplied charger to a wall outlet.

The Yomper is supplied with a charger compatible with a voltage range of 110 V to 240 V.

Only the type of mains plug may vary depending on your geographical area to ensure proper use:

- Europe: plug types C, E, F
- United States and Canada: plug types A, B
- Other regions: check that the power cable is compatible with local mains outlets. A plug adapter may be required.

Important: only use the charger supplied with the Yomper. If in doubt about plug compatibility, contact your dealer or a qualified professional.

While charging: the charger LED is **RED**. When fully charged: the LED turns **GREEN**.

For a fully discharged battery:

- Approx. 4 hours for the 24V 168Wh battery.
- Approx. 6 hours for the 24V 240Wh battery.

Essential Precautions :

- Never charge the batteries in the presence of or near flammable liquids or gases.
- Do not charge in locations where humidity may condense on Yomper components or its battery.
- Do not charge batteries at temperatures below 0°C (32°F) or above 40°C (104°F).

Charging the EasyGo Control Unit

1. Turn on the EasyGo unit.
2. Connect the magnetic charging cable.
3. A charging indicator will appear on the EasyGo screen.
4. Switch off the Yomper to prevent accidental activation.

Use only a charger with a maximum output of 5.0 V. Using a charger with a higher voltage will cause irreversible damage to the EasyGo control unit. Such damage is not covered by warranty.


3.7 Attendant throttle (Optional)

To prevent incorrect handling, we recommend connecting the attendant throttle only after the Yomper is securely docked to the wheelchair.

1. Attach the attendant throttle to the handle.
2. Connect the attendant throttle cable to the rear connector of the Yomper.

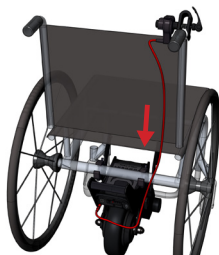3. Secure the control unit onto the backrest using the provided clip.

Once connected, **the attendant throttle overrides the EasyGo control unit** for speed adjustment.

Note: The third-party control can be used with or without the EasyGo control unit.

If the wired control is connected before powering on the Yomper, the EasyGo unit is not required to operate the device.

The maximum speed will then be limited to the maximum speed set in the EasyGo unit settings (see section 4.2).

4. Functions and Settings

4.1 EasyGo Control Unit Display

By default, the screen can display:

The current speed of the Yomper.

The battery level of the Yomper (YP).


The battery level of the EasyGo control unit (BTN).

To select which value to display (while the Yomper is stopped and the LED is flashing), press and hold one of the speed adjustment buttons.

4.2 Settings Menu

To modify the settings, the Yomper must be stopped or switched off.


Access the menu by pressing and holding the **far speed adjustment button** (the one furthest from you) for **more than 5 seconds**.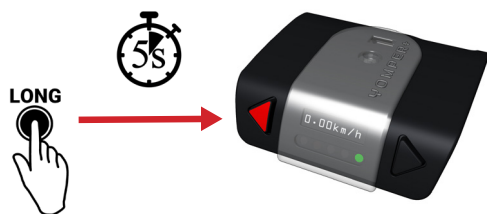


The menu operates as follows:

Confirm a selection: press the main button once.

Return to the menu: hold the main button for 2 seconds.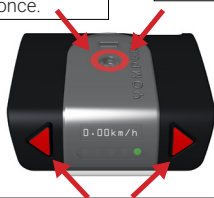

Navigate or adjust values: use the arrow buttons

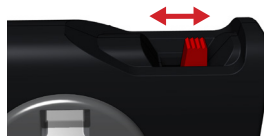Restart the EasyGo control unit (ON/OFF button) for changes to take effect.

Menu options:

- **Inside Speed:** Set the 5 indoor speed levels, within the limits configured by your healthcare professional.
- **Outside Speed:** Set the 5 outdoor speed levels, within the limits configured by your healthcare professional.
- **Acceleration:** Select the acceleration level (1 to 5, where 5 is the strongest).
- **Resistance:** Set the braking resistance when stopping the wheelchair in indoor mode (1 to 5, where 5 is the highest resistance).
- **Side Button:** Select left or right placement on the wheelchair (this reverses the « » buttons and the LED display direction).
- **Default Display:** Choose the default value shown on the screen.
- **Data:** Display stored Yomper data (total distance traveled, indoor/outdoor usage time, total usage count, and total Ah consumed since purchase or the last firmware update).
- **Scan:** pairing the EasyGo button with a Yomper.

Warning: if you enter the SCAN menu, the Yomper will lose its current pairing with the EasyGo.

The system will then ask you to enter the Yomper MAC address, which can be found on the certificate of conformity supplied in the Yomper case.

Note: The minimum speed setting must be at least 3.0 km/h (a higher value is possible).

How it works:

- Use « » to scroll through the menu.
- Press the main button to enter a menu option.
- Select the parameter to modify, adjust the value with « », and confirm with the main button.
- Restart the EasyGo control unit to apply the changes.

Example: Adjusting Speed Level 2 in Indoor Mode

- Power on the EasyGo control unit.
- Press and hold «» for more than 5 seconds to enter the main menu.
- Use «» to select « Inside speed ».
- Press the main button to confirm.
- Press «» to display : « Inside Speed, 2=XX »
- Press the main button to enter edit mode.
- Adjust the value using «» ou «» pour modifier la valeur
- Press the main button to confirm and save.
- Hold the main button for 2 seconds, then restart the EasyGo control unit.

4.3 Pairing

When using an EasyGo control unit for the first time, it must be paired with the Yomper. Have your Yomper compliance certificate ready.

- Switch on the Yomper, then power on the EasyGo control unit.
- In the menu, select SCAN, then confirm with the main button.
- Press the main button again to start the scan.
- The Yomper's MAC address will appear on the screen. Check that it matches the one on your certificate (use « » to select the correct address), then confirm with the main button.
- Restart the EasyGo control unit.

5. Maintenance

5.1 Inspections and preventive maintenance

Regular maintenance is essential to ensure the safety and longevity of the Yomper.

Recommended checks:

Monthly: Visually inspect the Yomper for any signs of wear or abnormalities.

Every 6 months:

- Check the condition of the tires and ensure they are inflated to 2 bar.
- Inspect and tighten the mechanical parts of the support.

Annually : Have the entire system checked by an **authorized dealer**.

Important: All technical interventions must be carried out by a qualified professional. Any unauthorized modifications or repairs may result in malfunctions or void the warranty.

5.2 Cleaning

The Yomper and EasyGo control unit require delicate cleaning to avoid damage to electronic components.

Cleaning guidelines:

- Use only a slightly damp cloth.
- Avoid any contact with water, as moisture can permanently damage electronics.
- Never use abrasive products, solvents, detergents, or aggressive cleaners.

Strict prohibitions:

- Never clean the Yomper with running water, a hose, or a pressure washer.
- Do not spray liquid directly onto the control unit or motor block.
- Never immerse electronic components in water.

Warning: Any damage caused by water infiltration will void the warranty and is the user's responsibility.

5.3 Transport

When transporting the wheelchair with the Yomper, follow these recommendations to prevent damage or accidental activation.

Preparation before transport:

- Always switch off the Yomper and the EasyGo control unit.
- Remove the Yomper and the control unit from the wheelchair.
- Use the dedicated transport accessories:
- Place the Yomper in its transport case.
- Store the EasyGo control unit in its protective pouch.

If the Yomper remains attached to the wheelchair during transport, ensure that it is **securely fastened and switched off**.

5.4 Materials and Protection

All materials used are corrosion-resistant and compatible with disinfectants:

- Aluminum chassis.
- Zinc-coated and stainless steel screws.
- High-strength plastics.

In case of heavy rain, **protect the EasyGo control unit** while driving (e.g., use a plastic "zip-lock" pouch).

5.5 Storage

If your wheelchair and the Yomper need to be stored for an extended period (several months), follow these recommendations:

Optimal conditions:

- Store your wheelchair according to the manufacturer's instructions.
- Keep the Yomper and its components in a dry place, away from moisture.
- Avoid prolonged exposure to direct sunlight (e.g., near a window).
- Prevent access by children or unauthorized persons.
- Charge the battery to around 80% before storage.
- For long-term storage, recharge the battery once a month to maintain its health.

5.6 Lithium Battery Maintenance and Safety

The Yomper and EasyGo control unit are equipped with lithium batteries, which offer **high autonomy and fast recharging**. However, these batteries require specific precautions to ensure safety and durability.

Built-in safety features

Our batteries are equipped with the following protection systems:

- Overcharge protection: Automatic stop when the battery is fully charged.
- Overheat protection: Thermal regulation to prevent excessive temperature rise.
- Short-circuit protection: Automatic cut-off in case of a detected short circuit.
- Over-discharge protection: Automatic shutdown to prevent deep discharge below a critical level.

Any attempt to modify or bypass these protection systems voids the warranty and may result in severe hazards (fire, explosion, irreversible battery damage).

Risks related to lithium batteries

- Fire or explosion risk due to excessive heat, short circuits, overcharging, or physical damage (shock, puncture, crushing).
- Toxic gas release in the event of severe failure.
- Performance loss if a non-approved charger is used.

Best practices for safe use

Inspection and Maintenance

- Regularly check the condition of the battery and charger: look for swelling, cracks, or corrosion on connectors.
- **Never use a damaged, deformed battery or one emitting an unusual odor.**

Safe usage

- Never expose the battery to extreme temperatures (below 0°C or above 45°C).
- Avoid moisture and direct water contact.
- Never leave the battery in a closed, hot environment, such as a car in direct sunlight.
- Do not disassemble or modify the battery, as this voids the warranty and creates immediate danger.

Safe Charging

- Use only the charger provided by the manufacturer.
- Never leave the battery charging unattended for extended periods.
- Unplug the charger once the battery is fully charged.
- Do not charge the battery in a humid environment, near flammable objects, or at temperatures below 0°C or above 45°C.

Storage

- Store the battery in a dry, well-ventilated area, away from direct sunlight.
- Keep the battery charge level between 40% and 60% if not used for long periods.
- If stored for more than one month, recharge it at least once per month to avoid deep discharge, which could cause irreversible damage.

Transport and Handling

- Always remove the battery from the Yomper during transport and place it in a secure case.
- For air travel, check applicable regulations on lithium batteries.
- Avoid shocks or drops that could damage the battery.

What to do in case of a problem?

Overheating or suspicious odor:

- Immediately switch off the device.
- Keep it away from flammable materials.
- Place it on a non-flammable surface (e.g., tile, concrete).
- Contact customer service before any reuse.

Damaged or deformed battery:

- Do not use or attempt to repair it.
- Contact your dealer for diagnosis and replacement.

Battery fire:

In the rare event of overheating or fire, never use water or liquids to extinguish the battery. Use sand or a Class D fire extinguisher, as recommended by battery manufacturers.

If the fire cannot be quickly contained, evacuate the area, ensure nearby people are safe, and call emergency services.

Important Warnings

- Never disassemble or modify the battery or charger.
- Never use a non-approved charger.
- Do not expose the battery to metal objects that may cause a short circuit.
- If in doubt, immediately contact customer support for assistance.


5.7 Battery Removal

When the Yomper is turned off:

1. Turn the Yomper upside down.
2. Unscrew the 4 screws of the cover.
3. Lift the battery to access the connectors.
4. Remove the connector lock and disconnect them.

Warning: Never attempt to open or modify the battery yourself.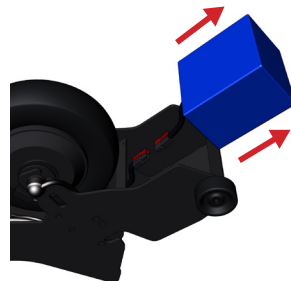

5.8 Reuse

If necessary, the Yomper can be fully removed (including its support) from the manual wheelchair, allowing you to restore the wheelchair to its original manual configuration without any structural modification.

After removal, the Yomper can be installed again on any other compatible wheelchair.

If your Yomper has been provided by your health insurance and you no longer need it, please contact your dealer. Your Yomper can be reused easily and cost-effectively.

Before each reuse, a full maintenance check of the Yomper must be carried out. The mounting brackets and adapter axles used to attach the Yomper components to your wheelchair can be easily and quickly removed from your old wheelchair and reinstalled on a new wheelchair by an authorized dealer or Acecare representative.

In addition to the cleaning guidelines provided in Chapter 5.2, before any reuse, all accessible plastic parts of the Yomper components must be disinfected. Use a cleaning product suitable for wipe disinfection, following the manufacturer's instructions.

5.9 Disposal

This device, its batteries, and its accessories are long-life products.

However, they may contain substances harmful to the environment if disposed of in locations (e.g., landfills) not designed for this purpose, as per the applicable legislation in the relevant country.

The «crossed-out wheeled bin» symbol (in accordance with the WEEE Directive) appears on this product as a reminder of the recycling obligation.

Respect the environment and deposit this product at the end of its life cycle at your local recycling center.

Please check the legislation in your country regarding waste disposal, as the WEEE directive for this product is not applied in all European countries.

The Yomper can also be collected by Acecare or Acecare dealers for disposal in compliance with environmental and legal requirements.

6. Troubleshooting

Before contacting your dealer, check the following points:

Issue	Solution
Yomper does not turn on	Check if a light appears after pressing the ON/OFF button. Verify the battery level and recharge your Yomper. Visually check the condition of the cables and the Yomper (without disassembling anything).
Yomper does not charge	Check the functionality of your power outlet. Verify that you are using the correct charger. Check if the charger LED changes from green to red during charging. Inspect the charging port on the Yomper. Check the battery gauge before and after 2 hours of charging to ensure progress. If the charger flashes red, unplug it and contact your dealer.
EasyGo control unit does not turn on	Check that the control unit switch [19] is in position I (ON position). If not, push the button to ON (towards the center of the button). Charge the EasyGo unit (wait approximately 10 seconds for the display to appear).
EasyGo control unit does not charge	Check that the control unit switch [19] is in position I (ON position). If not, push the button to ON (towards the center of the button). Charge the EasyGo unit (wait approximately 10 seconds for the display to appear).
EasyGo displays "Connecting" for more than 10 seconds	Check that the Yomper is turned on. Re-pair the Yomper and EasyGo (see 4.3 Pairing).
Yomper accelerates unexpectedly	If you are in a highly disturbed environment (e.g., strong medical imaging devices) or near an electromagnetic source, turn off the Yomper. The Yomper is EMC certified, meaning it is immune to normal everyday environments.
Yomper does not stop	In indoor mode, this can happen if the resistance level is set too high. Reduce the resistance in the settings. If pressing the main EasyGo button does not stop it, this is abnormal. Turn off both the Yomper and EasyGo and contact your dealer immediately.

Issue	Solution
Yomper stalls	This can happen if the slope is too steep, or if the wheelchair is too heavy or the tires are underinflated. Assist the Yomper manually or adjust its speed settings. This is a safety feature to prevent motor overheating. When the battery level is low (< 20%), the Yomper may also "stall" during heavy use. After a few seconds, you can resume operation. We recommend assisting the Yomper on steep inclines and recharging the battery as soon as possible. If you frequently travel on demanding routes or if the user weight is high, we recommend the optional 240Wh battery for better long-term performance. In indoor mode (green LED), excessive resistance can stop the motor (similar to braking on the hand rims). Switch to outdoor mode or increase the resistance.
Yomper detaches from its bracket	Once installed, check that the system is properly locked by pulling the Yomper firmly. Visually inspect the locking hooks to ensure they retract and reset correctly when pressing the release lever.
Loss of range	Battery range depends on several factors: • Total wheelchair weight, • Route incline, • Frequency of stops and starts, • Outside temperature (batteries can lose up to 40% capacity below 5°C), • Tire pressure. Poor battery storage conditions (see section 5.5 Storage) can significantly reduce lifespan and performance. If you notice a range loss under normal conditions, contact your dealer. Average battery lifespan is 3–5 years depending on use.
Mount detaches or makes noise	Check the proper tightening of all components. Contact your dealer if the issue persists.
When going over an obstacle with two wheels, the Yomper's wheel blocks the return to a seated position	The Yomper stopper is not properly installed. Contact your dealer immediately.
Bottom of Yomper scrapes during an obstacle	The mount is slightly too low. Contact your dealer to adjust the Yomper mount height to between 19 and 21 cm.

Serious Incident Reporting

In accordance with Regulation (EU) 2017/745 on medical devices, any serious incident occurring in relation to the device must be reported without delay to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.

7. Warranty

7.1 Warranty Coverage

Covered parts	Warranty period <i>(from date of purchase)</i>
Frame, motor, electronics, battery, and accessories	2 years

The expected lifetime of the complete product (excluding consumables) is 5 years from the date of sale.

To benefit from the warranty on a new product, you must keep the purchase invoice. This invoice will be required for any request to ACEKARE's after-sales service during the warranty period. Without the invoice, all repair costs will be charged to the customer.

ACEKARE commits to repair or replace free of charge the parts covered by the warranty, after verification by an authorized representative. The warranty applies only in case of material and/or manufacturing defects confirmed by ACEKARE.

Return shipping costs for the equipment are the responsibility of the buyer.

Beyond the warranty period, all repair costs, including parts and transport, will be charged to the buyer.

7.2 Warranty exclusions

The warranty does not cover the following:

- Wear parts: tires, bearings, casters, straps, etc.
- Damage caused by misuse: negligence, accidents, overloading, commercial use, malfunction, insufficient maintenance, or improper storage.
- Circumstances beyond ACEKARE's control (e.g., natural disasters, fires, floods).
- Modifications and/or repairs performed without ACEKARE's written approval.
- Defects caused by natural wear and tear or improper handling, particularly those resulting from non-compliance with the instructions in this manual, accidents, or damage caused by fire or water.
- Maintenance operations related to normal product use (e.g., tire replacement).
- Inspections carried out when no fault is found.
- External components and products used with the Yomper, particularly the manual wheelchair.

If the serial number on the frame has been modified, is not original, or does not match the warranty card, the warranty will be void.

The warranty is transferable from one private owner to another.

As the manufacturer, ACEKARE's responsibility is limited to repairing or replacing defective parts covered by the warranty.

To the extent permitted by applicable law, this warranty replaces any other written, oral, express, or implied warranties, including implied warranties of merchantability or fitness for a particular purpose. In case of conflict, these conditions prevail.

7.3 Limitation of liability

ACEKARE shall not be held liable for any bodily injury, material or non-material damage, whether direct or indirect, resulting from:

- use of the device contrary to its intended purpose;
- failure to comply with the safety and operating instructions provided in this manual;
- unauthorized installation, modification, or repair;
- use of accessories or components not approved by ACEKARE;
- insufficient or improper maintenance;
- negligence or misuse by the user or a third party.

ACEKARE shall not be liable where the device is used beyond the specified technical limits.

However, this limitation of liability shall not apply in cases of gross negligence, breach of mandatory legal obligations, or in circumstances governed by applicable legislation on liability for defective products.

The user remains responsible for ensuring that the device is suitable for their personal use and for complying with applicable local regulations.

8. Technical Specifications

8.1 Performance

Specification	Value
Estimated range	23 to 35 km depending on the battery
Maximum speed (flat surface)	10 km/h
Maximum speed (3% slope)	4 km/h
Motor power	250 W
Motor voltage	24 V
Operating temperature	-25°C to +50°C
Theoretical consumption in straight line	5,9 Wh/km
Theoretical consumption during maneuvers	22,5 Wh/km

Important note on range:

The stated battery range (23 to 35 km) is an estimate based on tests carried out under optimal conditions, in accordance with ISO 7176-4: 2008:

- Passenger weight: 100 kg
- Moderate ambient temperature (~20°C)
- Flat surface with good quality pavement
- Constant and moderate speed (no abrupt accelerations)
- No additional load (accessories, luggage, etc.)
- Use of high-pressure performance tires (e.g., Schwalbe RightRun 25-540)

Factors affecting actual range:

Several parameters can impact the distance achievable with a full charge:

- User and extra equipment weight: A heavier load requires more energy.
- Type of terrain: Rough surfaces, gravel, slopes, or curbs increase energy consumption.
- Driving style: Sudden accelerations and frequent stops reduce range.
- Outside temperature: Lithium-ion battery performance decreases in cold or very hot weather.
- Wheelchair tire pressure and type: Underinflated tires increase rolling resistance and energy consumption. Using solid tires can almost double energy consumption compared to high-performance high-pressure tires (e.g., Schwalbe RightRun 25-540).

Tips to optimize range:

- Maintain a steady and moderate speed.
- Avoid sudden accelerations and braking.
- Store and use the battery within recommended temperatures.
- Regularly check and maintain the wheelchair's battery and tires.

ACEKARE cannot be held responsible for a range lower than the estimated value if the actual usage conditions differ from the test conditions.

8.2 Dimensions and Weight

Specification	Value
Overall length	391 mm
Overall width	164 mm
Overall height	335 mm
Weight	4,9 kg
Wheel diameter	8 inches (203 mm)
Battery type	Lithium-ion
Nominal voltage	24 V
Minimum voltage (discharged)	19 V
Maximum voltage (charged)	29,4 V
Overall dimensions	101 mm (L) x 101 mm (W) x 70 mm (H)
Total weight of Yomper (168 Wh)	5,8 kg
Total weight of Yomper (240 Wh)	6,2kg
Maximum user weight	150 kg

8.3 Compatibility

Rigid frame wheelchair

Element	Compatible dimensions
Wheel diameter	24, 25, 26 inches
Camber bar diameter	15 to 40 mm
Under-seat bar diameter	22 to 59 mm

Folding frame wheelchair

Element	Compatible dimensions
Wheel diameter	22, 24, 25, 26 inches
Seat width	32 cm to 55 cm

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