



FLEXLIFE **D9**
BY DECON







 **FLEXLIFE** **D9**
BY DECON

Freedom

on four wheels



FLEXLIFE D9 Basic & Comfort

Feel free
with
FLEXLIFE!

WHAT DO I GET WHEN I ORDER FLEXLIFE?

- Flexlife
- Choose between Basic or Comfort
- High seating comfort
- Low weight 26,5kg
- Strength for difficult slopes, 10 degrees
- Tiltable backrest
- Smart battery system, flight approved
- Large selection of accessories
- Possibility for different seating positions
- Battery and battery charger
- Manual
- 2 years warranty



Flexlife Basic



Flexlife Comfort

For those who have difficulty walking or use a wheelchair/electric wheelchair for everyday use but want to be more flexible - here is a convenient option for you to be able to take the electric wheelchair with you on your travels and everyday activities.

The folding electric wheelchair Flexlife offers two different equipment levels Basic and Comfort, no users are the same. They are lightweight (from 26.5kg incl. battery), comfortable, stable and reliable. The smart battery systems allow you to easily make your travel flexible.

A number of smart accessories mean you can get a personal flexlife. Equip your Flexlife with, for example, mobile phone holders, calf rests, headrests and more.



FLEXLIFE

ARTIKELNUMMER

Flexlife D9 Basic

FLD09B

Flexlife Basic: Bima standard seat cushion, backcushion water-repellent fabric, tiltable backrest in 5 positions, adjustable and foldable footplate, 8" frontwheels, 12" massive rear wheel, quick-release for batteries, Anti-tippers with quick-release, locking for the chair in folded position, small shopping bag

Flexlife D9 Comfort

FLD09C

Flexlife Comfort has the same standard equipment as Flexlife Basic with upgrade to: Bima Kombi seat cushion (pressure relieving), lighting kit, swing-away mount for joystick, leg support, armrest pad

FLEXLIFE D9 Basic



1 Lock the chair in a folded position. The latch is stored under the footrest.

2 Five back straps for adjustment
Padded back in water-repellent fabric from Bima
Tiltable backrest with 5 options

3 Anti-tippers with quick release, easy to remove

4 Solid tires, do not need to be pumped and no risk of puncture

5 Attachments for secure in a car during transport

6 The footplate is adjustable in height and can be folded in under the chair

7 Double batteries with quick-release and battery indicator, can be charged separately

8 Bima standard seat cushion

9 Joystick for right- or left mounting
Charging socket for the batteries

FLEXLIFE D9 Comfort



Flexlife Comfort contains everything included in Flexlife Basic with the following changes/additions:

1 Padded armcushions

2 LED-lightning back

3 Leg support, easy to remove

4 LED-lightning front

5 Bima seat cushion Combi, pressure relieving

6 Swing-away-mounting for joystick

FLEXLIFE D9 Basic & Comfort

TECHNICAL DATA

Weight incl. batteries	26,5 kg (23,5 kg exkl. batteries)
Speed	6 km/h
Range*	25 km
Motor	2x250W brushless
Joystick	Right, left or mounted as assistant controller
Battery	2 litium batteries à 6 Ah, 24V
Battery power**	288 Wh (flight approved)
Battery weight	1,5 kg battery
Charging time	6-8 charging (100% laddning)
Turning radius	83 cm
Backwheel	12" (32 x 5,7 cm) solid backwheels
Frontwheel	8" solid fronwheels
Seat height without cushion	48 cm
Seat width	47 cm (with widening kit 54cm, accessoires)
Seat depth	43-48 cm
Max. userweight	150 kg
Fabric	Bima, waterresistant fabric
Tiltable backrest	5 positions, from 90 to 115 degrees
Adjustable footrest	Height adjustable in two positions, foldable
Dimensions upfolded in cm	93L x 59W x 98H
Dimensions folded in cm	58L x 35W x 78H
Warranty	2 years

*may vary depending on surface, user weight and weather

** Flight-approved batteries, max. 300 Wh is approved for air transport



FLEXLIFE D9 Basic & Comfort

Batteries & Charger

Battery	Articlenumber
---------	---------------

Standard battery D9 24V x 6 Ah	FLD09-2005
--------------------------------	------------

Battery charger	Articlenumber
-----------------	---------------

Standard battery charger	FLD09-2006
--------------------------	------------

Battery charger adapter	
-------------------------	--

Accessoires

Tillbehör

- | | |
|------------------------|-------------------------|
| 1. Leg support | 6. Attendant controller |
| 2. Kombi Cushion Bima | 7. Swing away mounting |
| 3. Mobile phone holder | 8. Widening kit to 54cm |
| 4. Head support | 9. Bottle holder |
| 5. Transportbag | 10. Armrest increase |



1



2



3



4



5




6



7



10

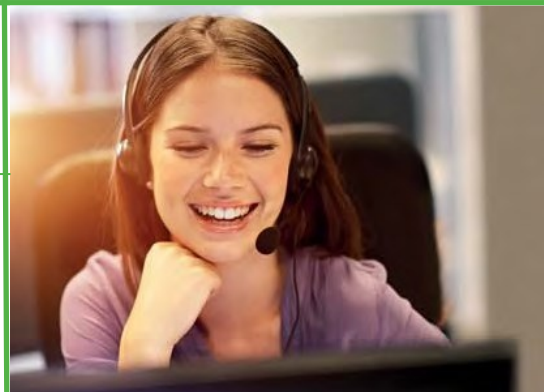

**DO YOU HAVE ANY QUESTIONS OR
WANT TO ORDER?**

0345-40880

info@decon.se

We'll be happy to help!

Checklist when ordering

- For standard equipment see "WHAT DO I GET WHEN I ORDER FLEXLIFE"
- Check if there is any accessory desired
- Contact us if you can't find what you're looking for
- Check your order confirmation, if something is wrong or unclear contact your convenience store or Decon immediately

FLEXLIFE Quick start guide



1. Open the box carefully with the help of a knife, put down the box and remove the styrofoam.



2. Remove the plastic bag and transport covers.



3. Think about the environment and recycle plastic and styrofoam in recycling.



4. Easily fold up your Flexlife by grabbing the back and pulling it up and back.



5. The locking hook automatically attaches to folding. Before first use, we recommend that you check that the locking hook is hooking in.



6. Mount on the anti-tipper. Pull the quick release and place the tippers in the existing hole, release the quick-release. You can easily remove the anti-tipper if necessary.

FLEXLIFE Snabb Start Guide



7. Remove the batteries and insert them into the frame of the chair. You can easily remove the batteries using the quick release. If you want to charge the batteries separately, you can use the supplied adapter cable.



8. Mount the joystick.
The joystick is in the bag under the chair. Take it out and place it in the hole at the armrest. Pull the screw on.



9. Carefully assemble the connectors on the joystick cable.

Note! Make sure that the white markings match the cable before you put them together, do not force the cable. Screw the plug together.



10. Insert the included seat cushion.



11. Pull the red levers down to connect the engines.

Note! Use hand power only.



12. Turn on the joystick



Your flexlife is now ready to drive!



Dealer information:



Decon Wheel AB Södra Ekeryd 119 31493 Hyltebruk

Phone: 0345 - 40880 Fax: 0345 - 40895 E-mail: info@decon.se www.decon.se

Decon reserves the right to make any changes made after the time of this printing. All rights are reserved for Decon.
Subject to printing and typing errors, as well as expired goods and incorrect images.