
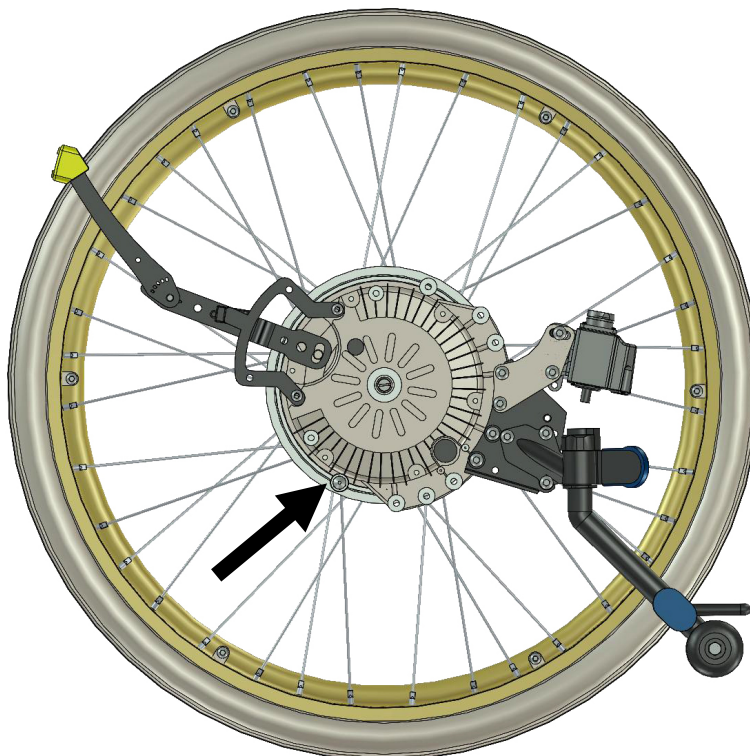



decon wheel

Assembly instructions for Exigo 20 / Legend 2 Exigo
MED3055


1. Remove wheels and hubsleevers on both sides.

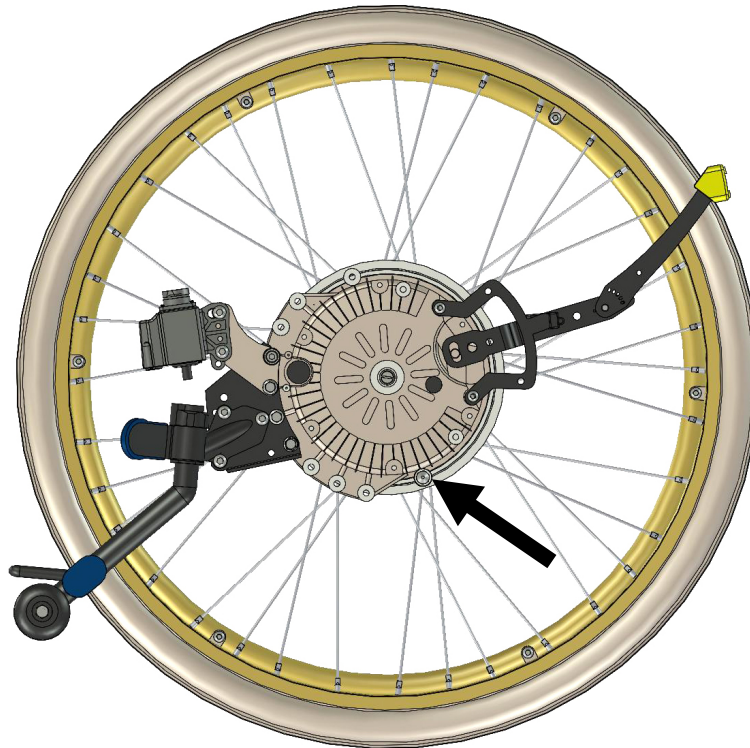

2. Check the correct placement for the stop pin on the E-drive unit.

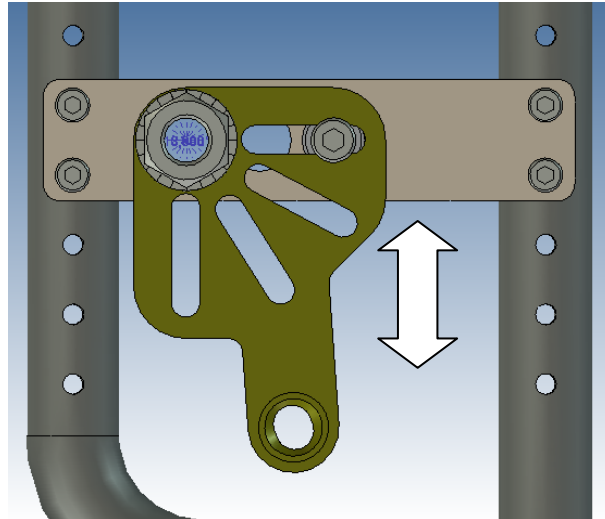

3. Placement for the adapter plate.

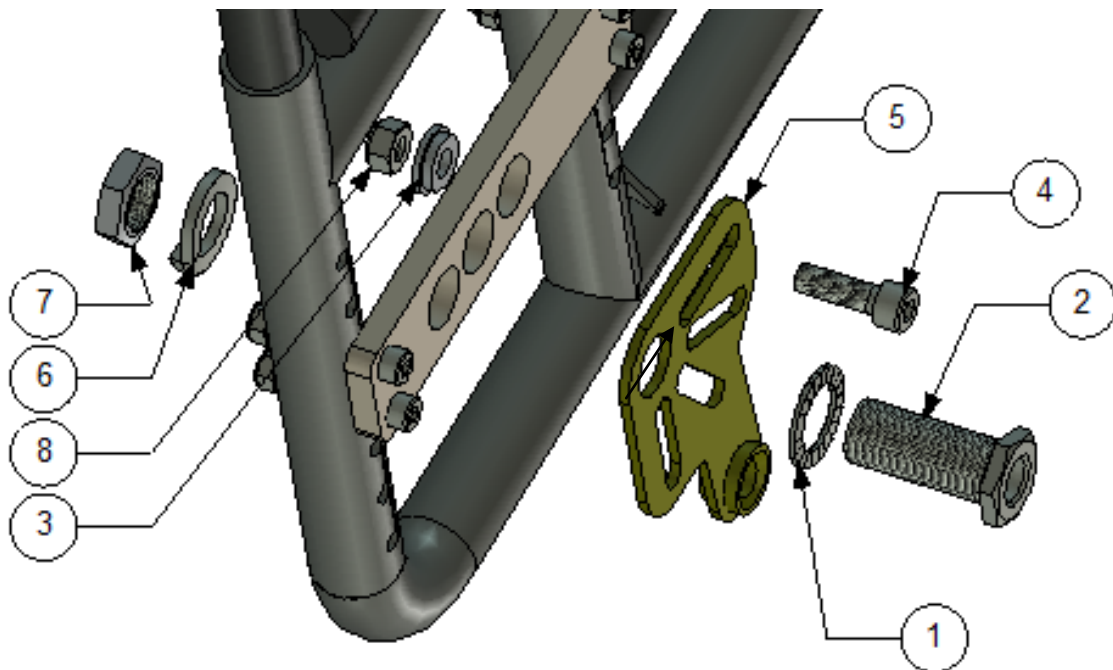


4. Assembly the plate (right side) according to picture.

5. Parts pair mountingset:

Item No.	Part Number	Qty	Description
1	405241	1	Nordlock Washer
2	HW084	1	Hubsleever
3	100 400 007	1	Spacer
4	XSM8X25	1	Screw M8 x 25
5	MED 2004	1	Bracket
6	XSM18FJ	1	Single Coil Washer M18
7	XSM18M	1	Nut M18x1
8	XSM8ML	1	Locknut M8

6. Assembly the back placed battery bag.

Exigo 20 with joystick

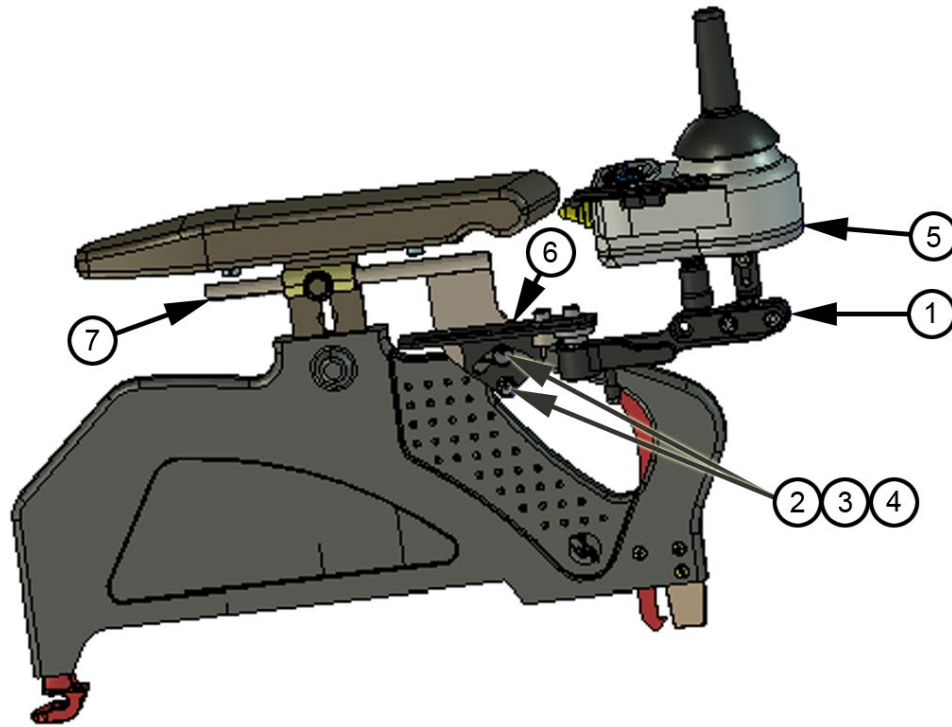

Item No.	Part Number	Qty	Description
1	XOC-48160-B0	1	Upper holder assembly 2
2	XSM6X16	2	Screw Round M6 x 16
3	XSM6BR	2	Washer Plain M6
4	XSM6ML	2	Nut Self-Locking M6
5	XOC-48110-A0	1	Remote Control Assembly
6	XOC-46295-00	1	Sheet Metal Bracket
7	MED1328	1	Joystick Bracket