


**Assembly instructions for Ottobock Dino 3
MED3022**

1. Remove wheels, brakes and brake handle on both sides.



2. Check the correct placement for the stop pin.

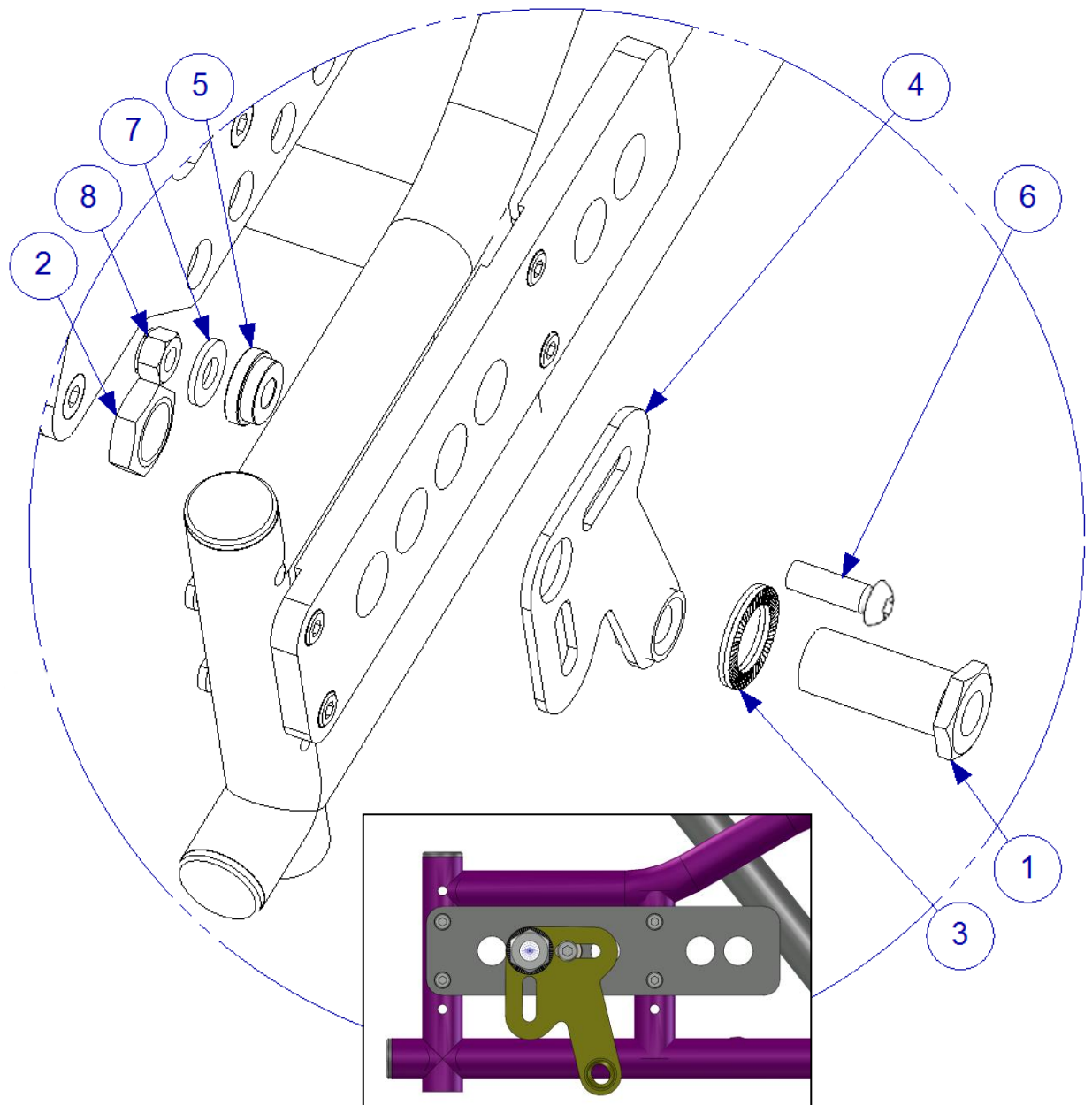


3. Assembly the plate (right side) according to picture.


Parts pair mountingset:

Item No.	Part Number	Qty	Description
1	HW084	1	Sleeve
2	XSM18M	1	Nut M18x1
3	405241	1	Nordlock Washer M18
4	MED2000	1	Bracket
5	100 400 008	1	Spacer
6	XSM8X25	1	Screw M8x25
7	XSM8BR	1	Plain Washer
8	XSM8ML	1	Locknut M8


4. Assemble the back placed battery bag.

MED1347

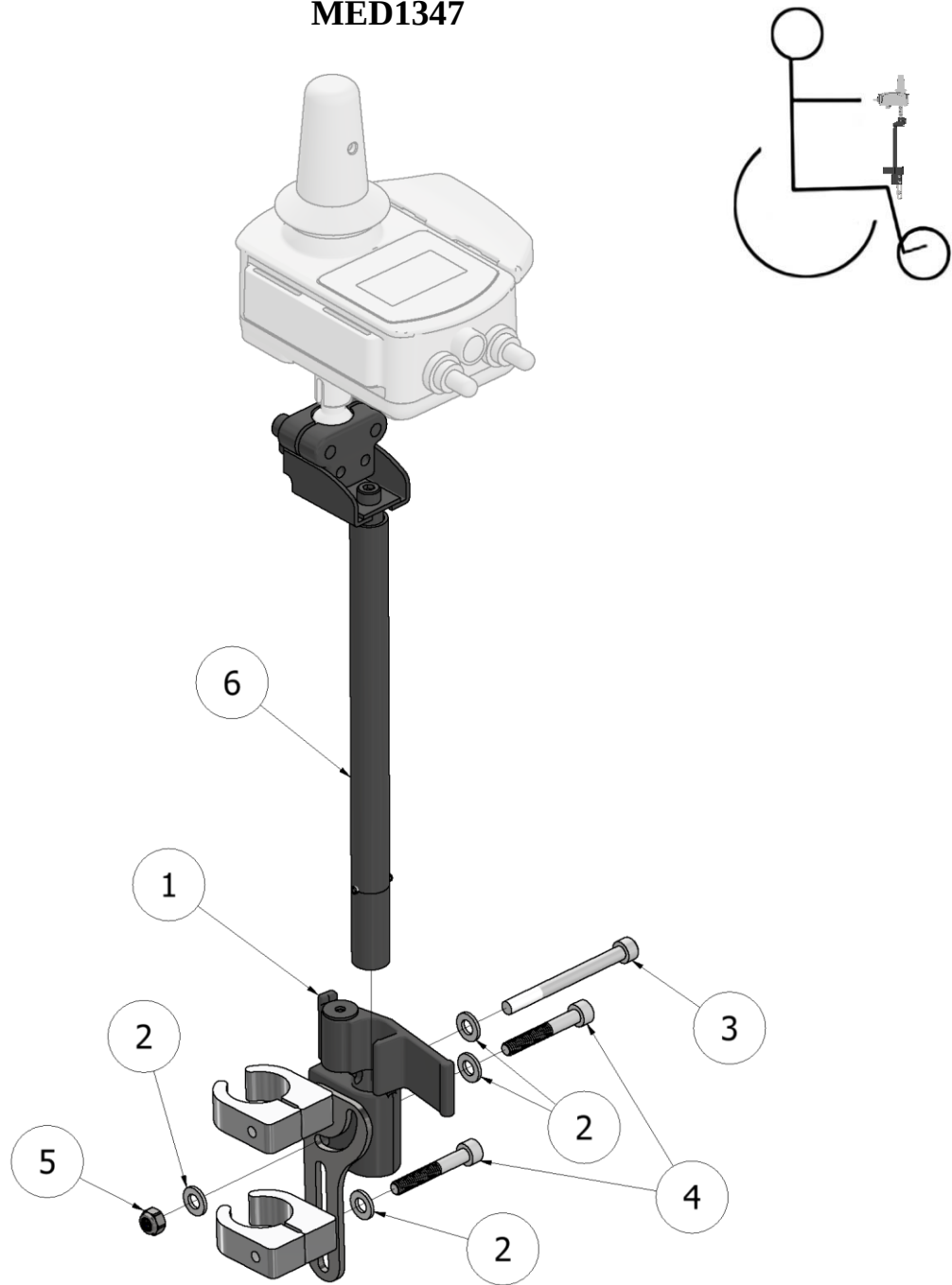

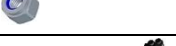1	XOF-48180-A0	1	
2	XSM6BR	4	
3	MC6SM6X65	1	
4	MC6SM6X40	2	
5	XSM6ML	1	
6	XOF-48160-00	1	


Assembly of the Attendant Control

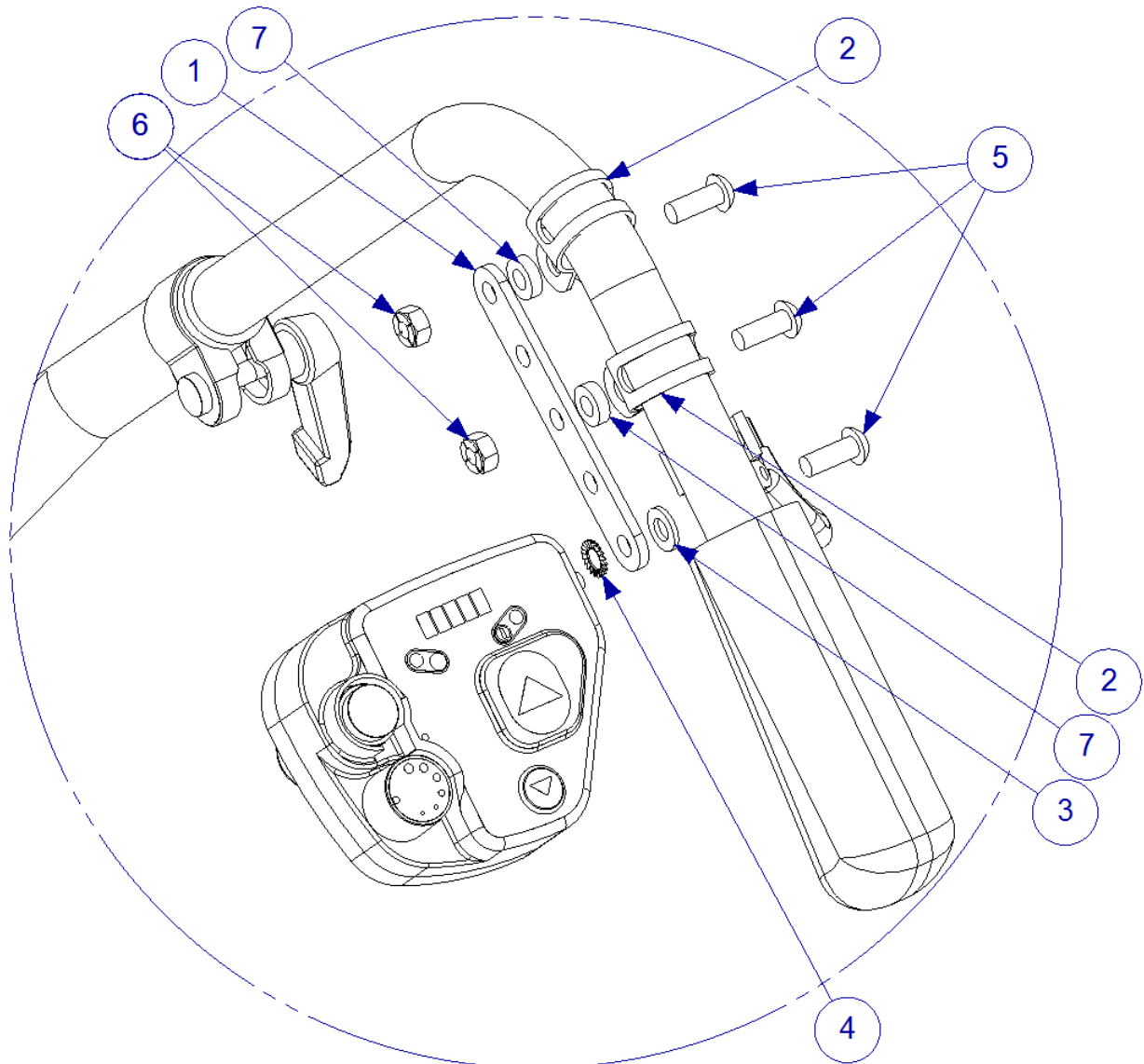

Item No.	Part Number	Qty	Description
1	MED1317	1	Bracket
2	XClamp2	2	Clamp
3	XSM6BR	1	Washer Plain M6
4	XSM6BRTG	1	Tag Washer M6
5	XSM6X16	3	Screw Round M6 x 16
6	XSM6ML	2	Nut Self-Locking M6
7	100 400 025	2	Spacer 12x6x3,6



M3	1,2 Nm
M4	3 Nm
M5	6 Nm
M6	10 Nm
M8	25 Nm
M10	50 Nm
M12	85 Nm

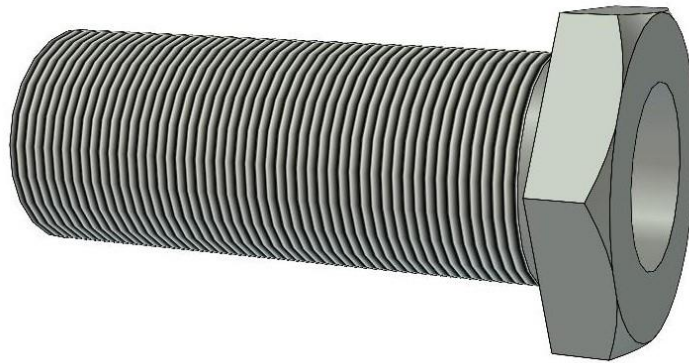

M16x1	30 Nm
M18x1	40 Nm