



1. Remove wheels and original hub sleeves with the brakes.

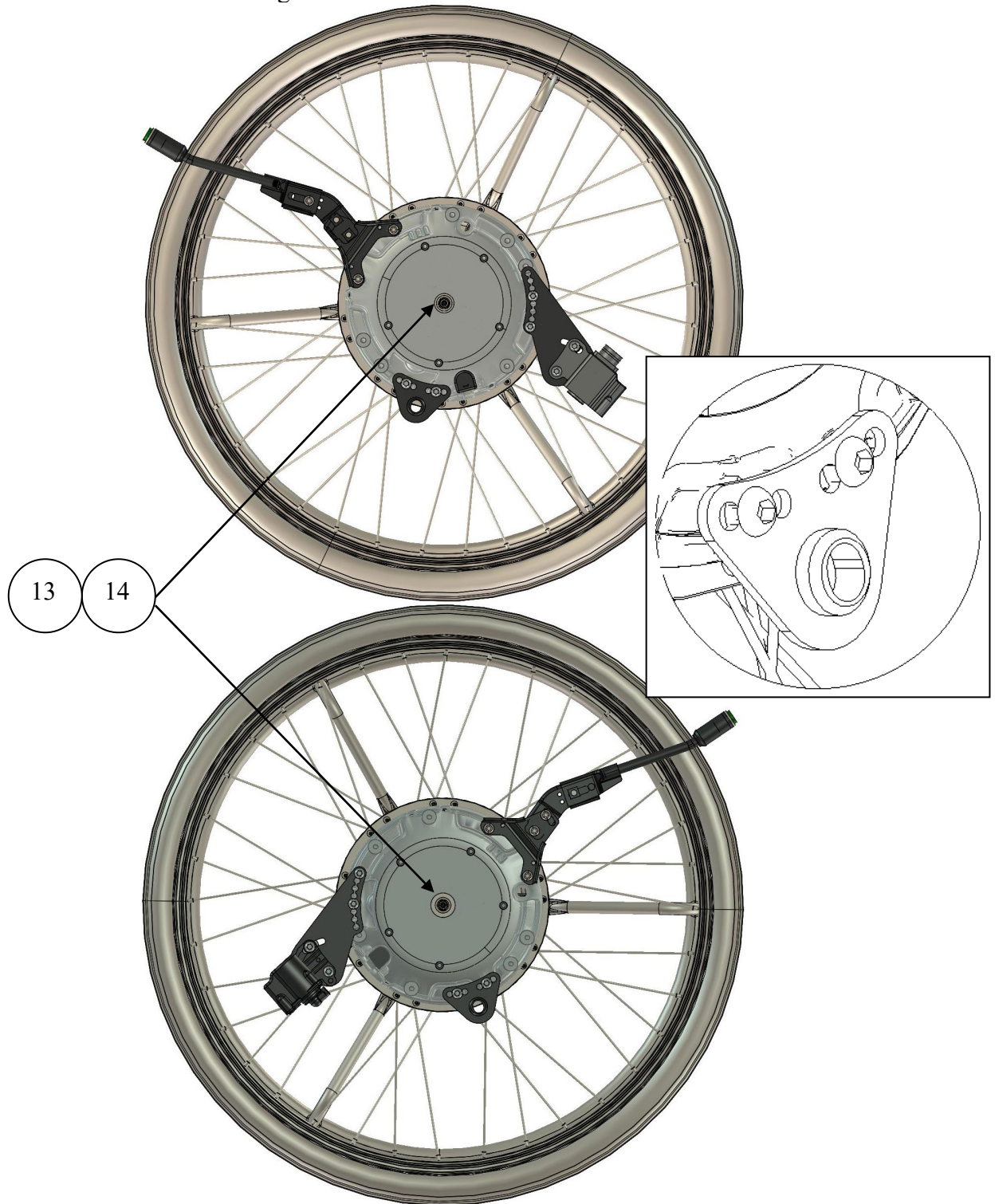

2. Check the correct placement for the eMove, and that it is the axle $\varnothing 12,0 \times 51$ with Part Number: XOE-2530E-BO and Spacer with Part Number: XOE-25383-00.

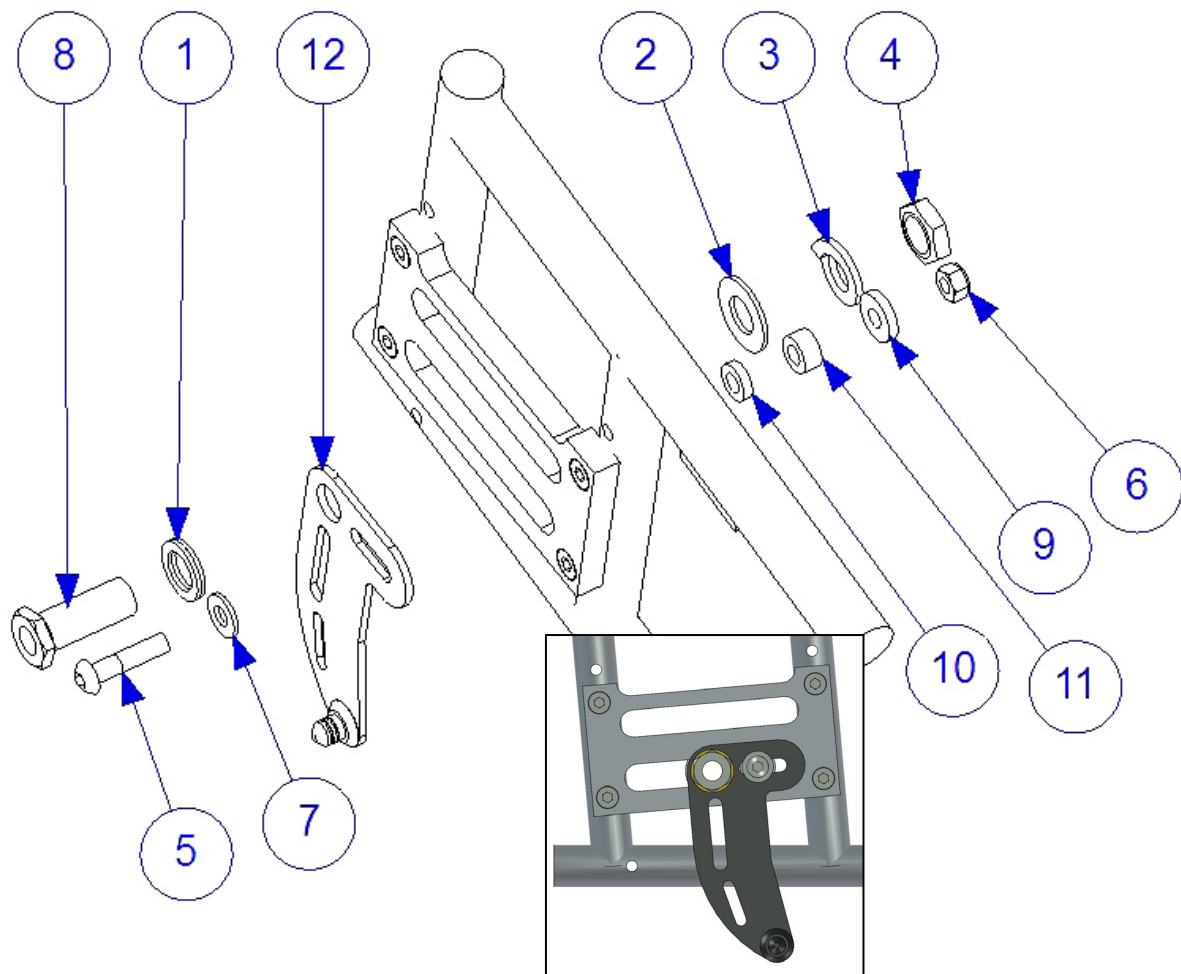

Parts pair mounting set:

Item No.	Part Number	Qty	Description
1	405239	1	Nordlock Washer
2	XSM16BR	1	Plain Washer M16
3	XSM16FJ	1	Single Coil Washer M16
4	XSM16M	1	Nut M16X1 Low
5	XSM8X35	1	Screw Round M8 x 35
6	XSM8ML	1	Lock Nut M8
7	XSM8BR	1	Plain Washer M8
8	HW063	1	Sleeve
9	00876-08-00	1	Washer 8,4x21x4 DIN7349
10	100 400 001	1	Spacer
11	100 400 003	1	Spacer
12	XOE-22294-BO	1	Right Bracket
13	XOE-25383-00	1	Spacer 12
14	93440-120520	1	Circlip

Note! If you only order the mounting set, the axle, axle spacer and the circlip **are not included**, for replacement see the servicemanual.

3. Assemble the back placed battery bag.