









Meyra - SMART f/s
MED3175

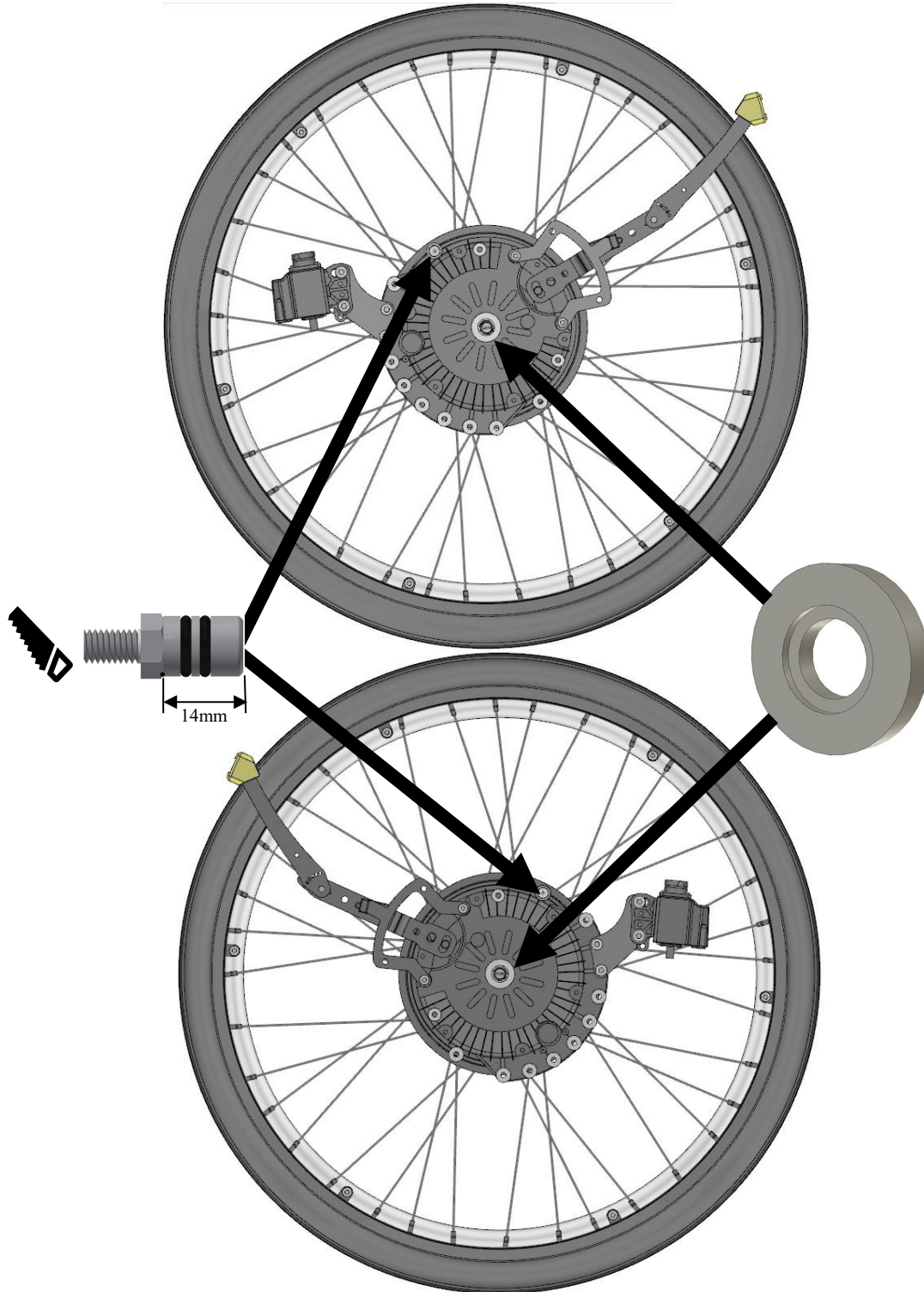

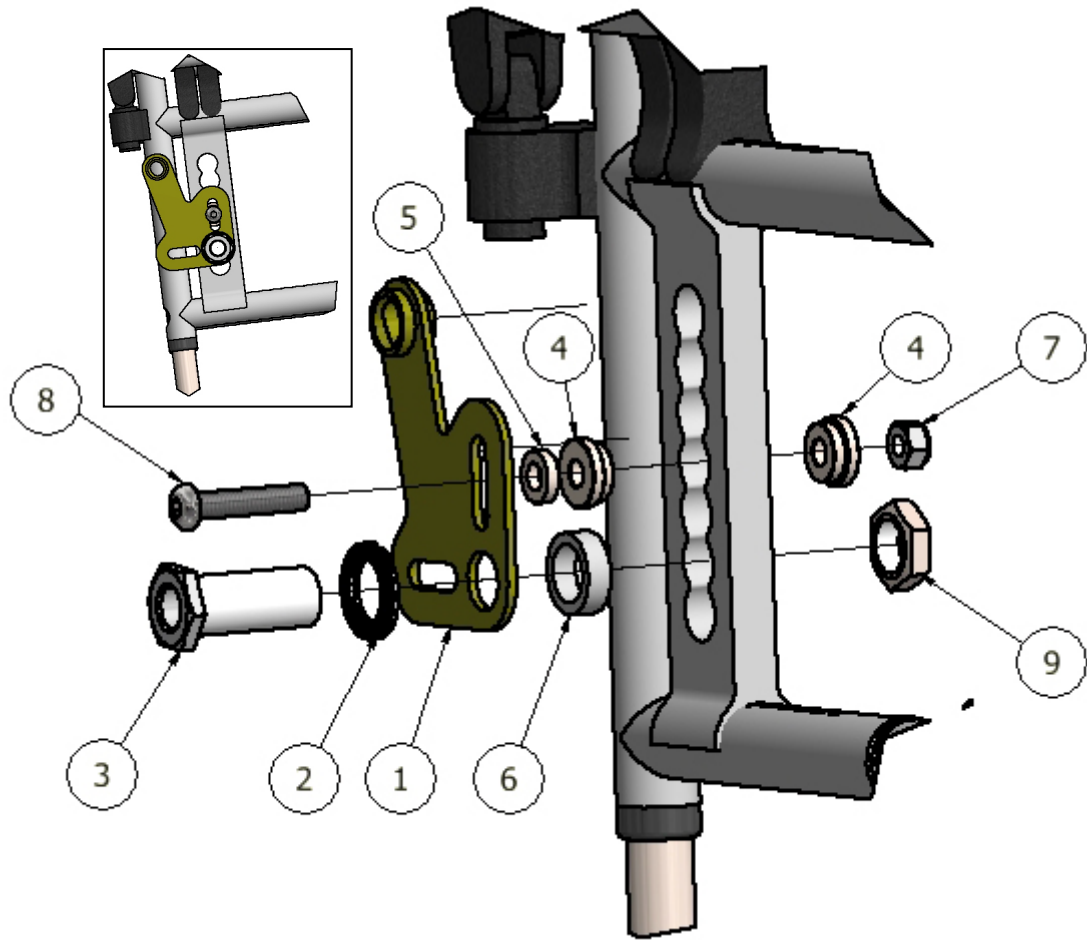


1	1	MED2000	
2	1	405241	
3	1	HW084	
4	2	100 400 008	
5	1	100 400 001	
6	1	100 400 016	
7	1	XSM8ML	
8	1	XSM8X40	
9	1	XSM18M	

Code 81

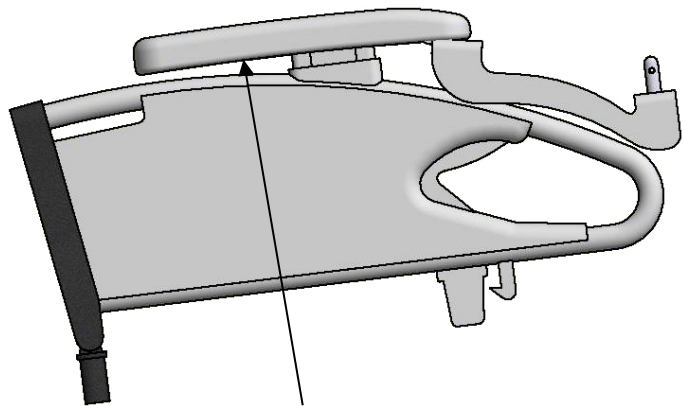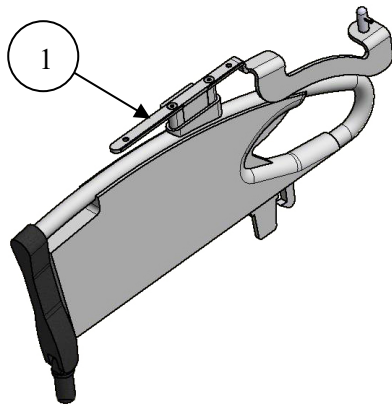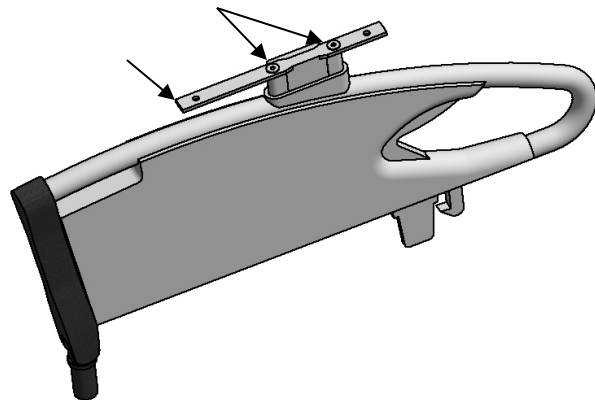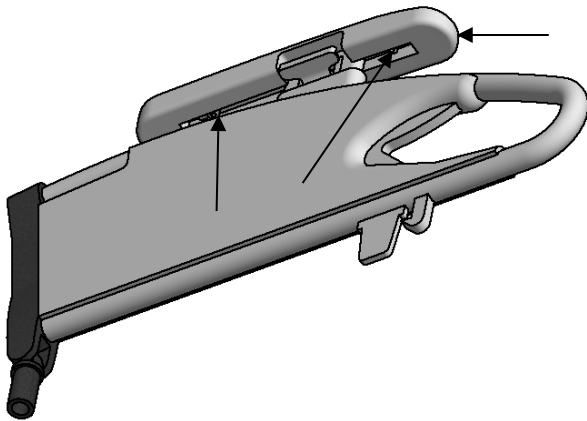


MED1428L/R

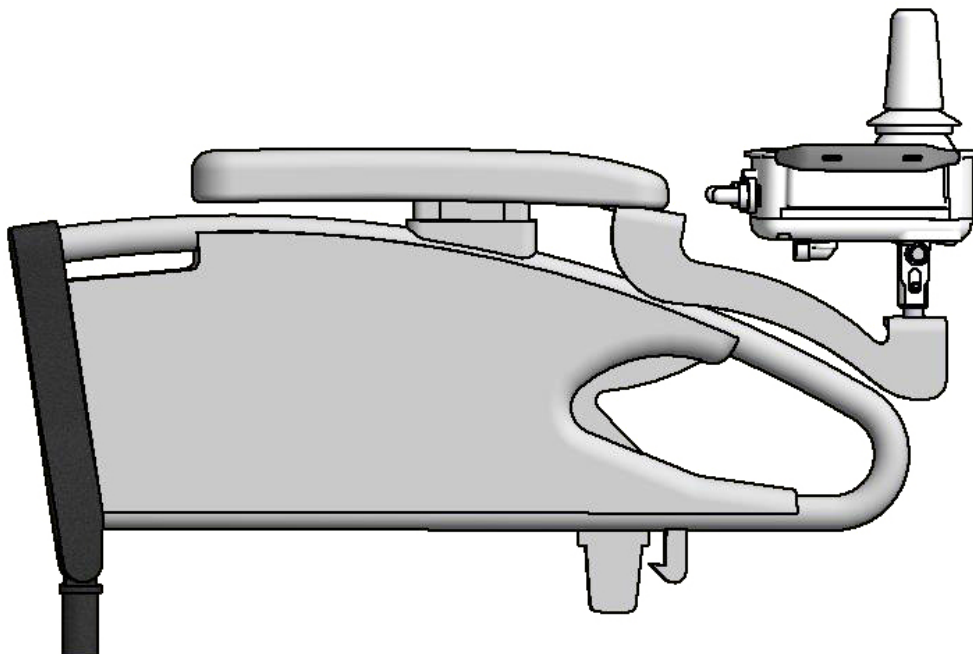

1	MED1428R	1	
---	----------	---	--

 = MED1428L


Code 21, 84 & 748
MED4054R

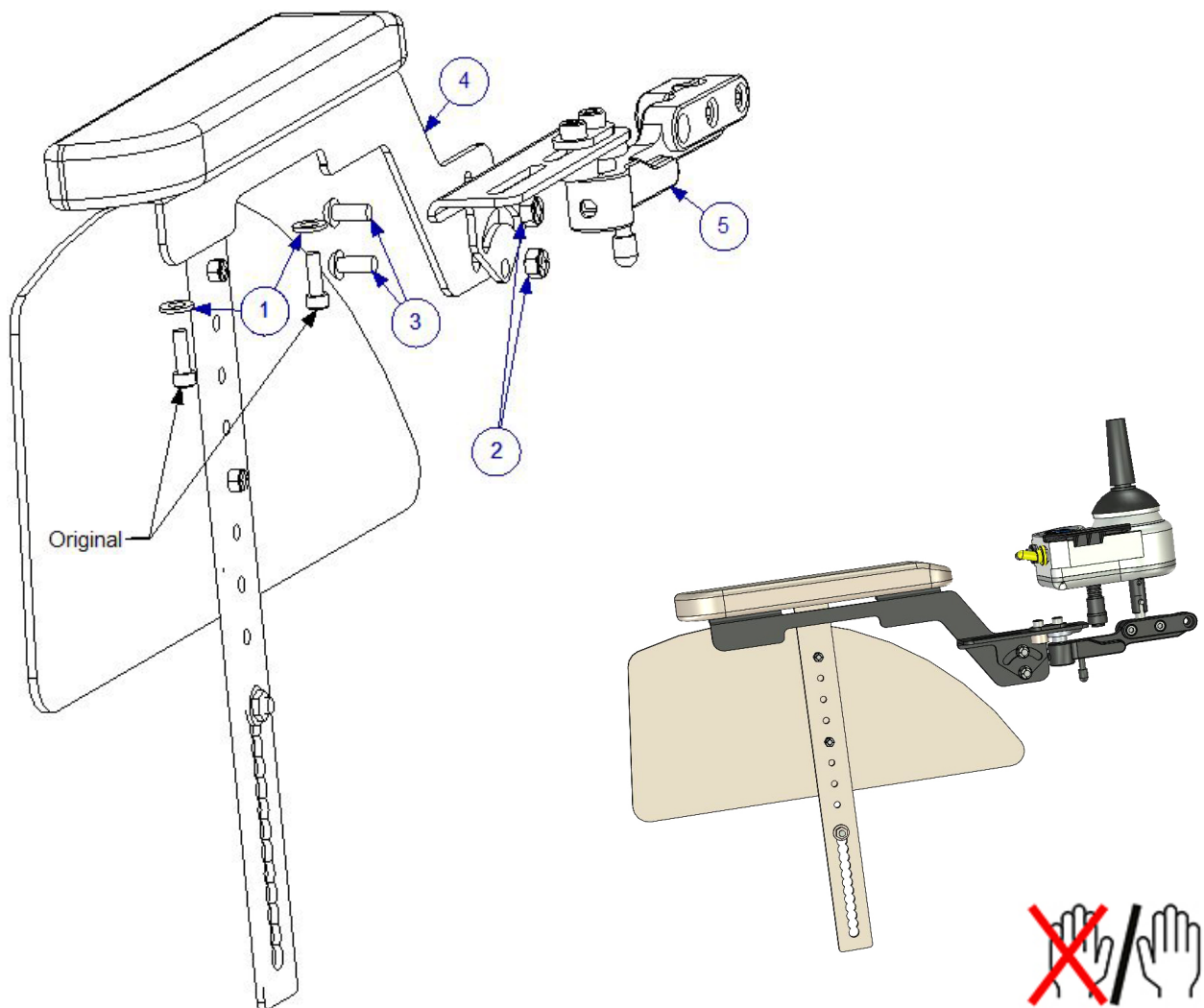

1	XSM6BR	2	
2	XSM6ML	2	
3	XSM6X14	2	
4	MED1500	1	
5	XOC-48160-B0	1	

 = MED4054L


Code 21, 84 & 748
MED4055R

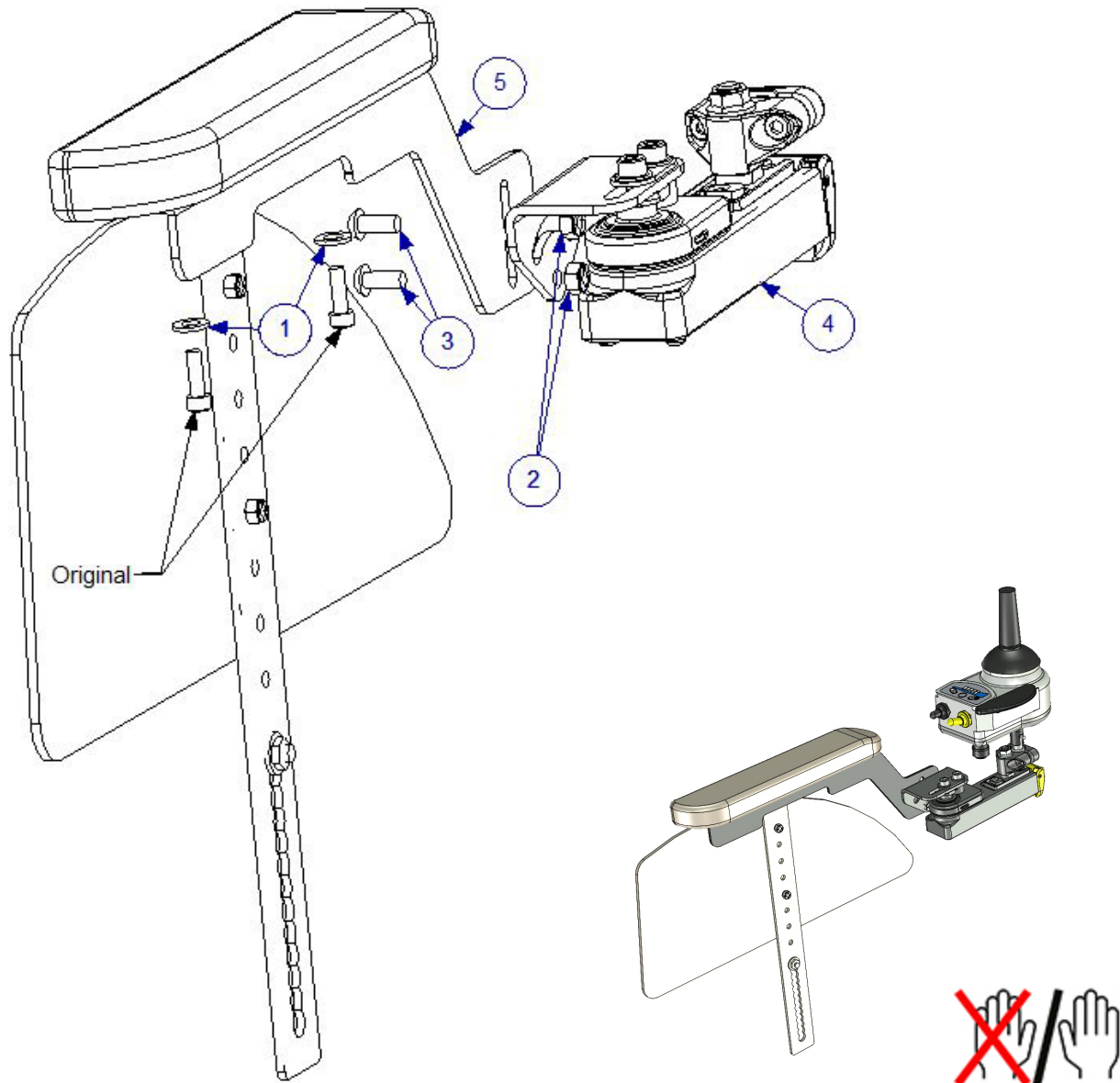

1	XSM6BR	2	
2	XSM6ML	2	
3	XSM6X14	2	
4	X0F-48160-D00	1	
5	MED1500	1	

 = MED4055L

MED1218

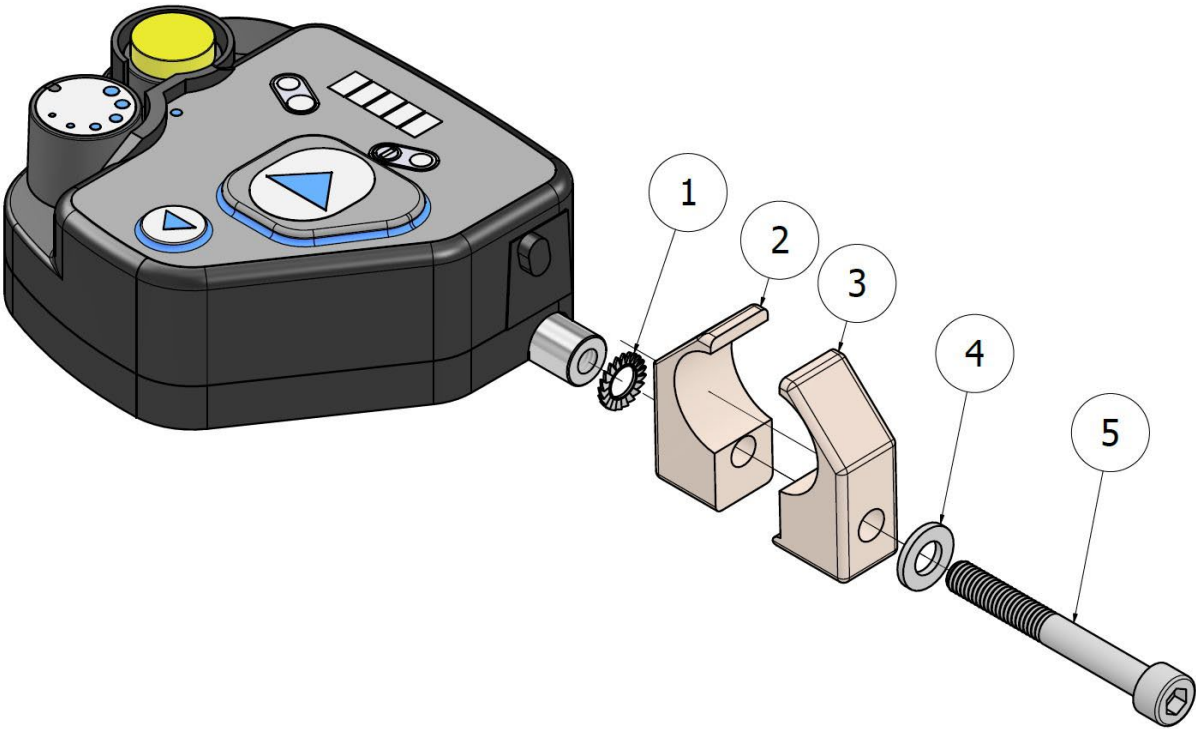


1	XSM6BRTG	1	
2	XOD-2612K-00	1	
3	XA2-2612E-00	1	
4	XSM6BR	1	
5	MC6SM6X50	1	



MED1218 + MED1643




MED1719
SMARTf

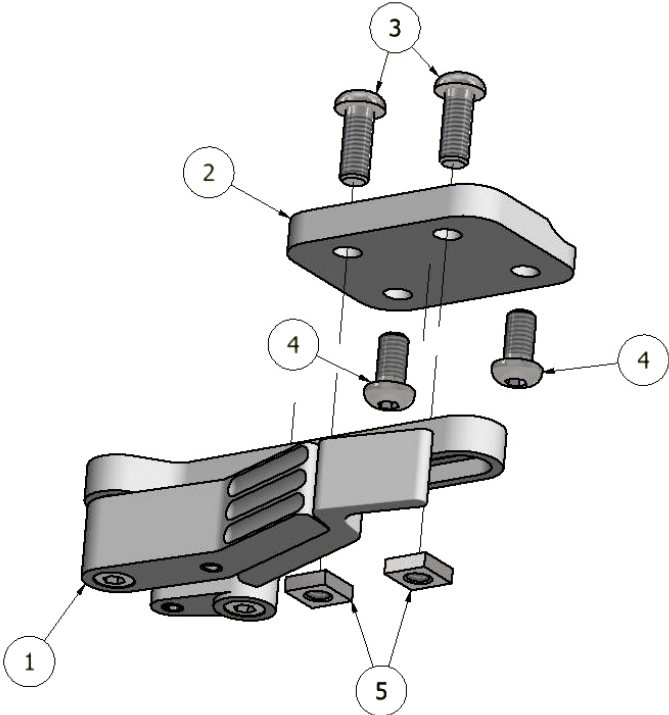

1	Original	1	Original NOT INCLUDED
2	310047	1	
3	XSM6x16	2	
4	XSM6x12	2	
5	AS 1474 - M6	2	



M3	1,2 Nm
M4	3 Nm
M5	6 Nm
M6	10 Nm
M8	25 Nm
M10	50 Nm
M12	85 Nm

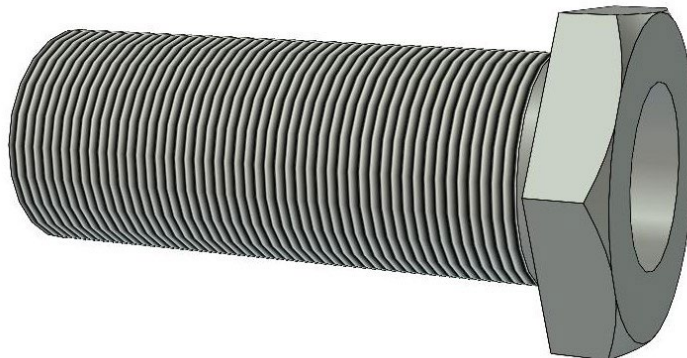

M16x1	30 Nm
M18x1	40 Nm